

# The original BodyArmor® CastShoe™ for cast bandages.

Easy to use!



- > rear-entry by flapping the shoe heel backwards – unique bungee closure system patent registered
- > secure hold of the heel
- > toe protection
- > shock absorbing and ultralight EVA material
- > stain resistant, hygienic surface for easy cleaning
- > washable
- > right and left side

**DARCO**



## Instructions for use

1. Unhook the bungee closure.
2. Push the button on the bungee closure toward the toes to unlock and loosen the ankle strap.  
Flapping the shoe heel backwards helps inserting the foot into the shoe.
3. Pull and tighten the ankle strap for closure.
4. Hook the bungee closure again. Done!

Item-No.	Color	Size	Inside shoe length
----------	-------	------	--------------------

CS0	Dark grey	XS	24,0 cm
CS1	Dark grey	S	26,5 cm
CS1	Dark grey	M	29,5 cm
CS3	Dark grey	L	31,0 cm
CS4	Dark grey	XL	34,5 cm

## Please ask for further information:

... tick appropriate box and send us this flyer  
by fax +49 (0)88 07.92 28-22 or send an e-mail to [info@darco.de](mailto:info@darco.de)

Please send me further information.

Please contact me.