## USA/GB To the patient

Your physician has dispensed to you a lower leg walking cast replacement. The design of the rocker sole warrants a physiologically correct gait by means of the »rolling-off« movement. Walking with the Body Armor® Walker II is more difficult than walking with conventional footwear and its correct application is essential for the walker's functionality – please read the following precautions.

### Caution

- > Do not attempt to drive while wearing this walker.
- > Use extreme caution when climbing stairs, stepping up onto curbs and walking on uneven surfaces.
- > Walk more slowly than usual. You are not wearing a normal street shoe and more clearance is required for the wedge sole.

The application of this aid should be supervised by a physician or healthcare specialist. In case of pain, swelling or skin irritation please call a doctor immediately! Avoid excessive compression of the air bladders!

Caution should be used on patients with neuropathy to ensure that the walker does not restrict blood circulation. Patients should call their physician immediately, if any tingling, burning, numbness or foul odor becomes noticeable.

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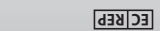
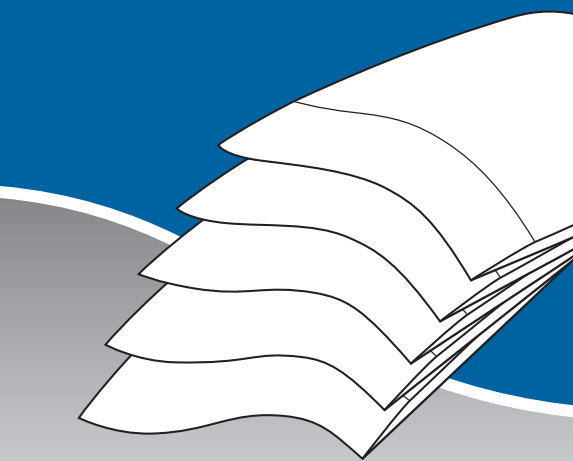
for Body Armor® Walker II

# DARCO

**ATW™ Achilles Tendon Wedge**  
for the Body Armor® Walker II  
(left/right)

The achilles tendon wedge set comprises heel wedges of different sizes. 3 wedges can be used one upon the other for perfect adjustment to shape and length of the individual foot, which may then be fixed in the heel area of the Walker.

For the Body Armor® Walker II of DARCO you may usually use the 3 smallest wedges for »XS« size, the three medium wedges for »S« size and the three largest wedges for »M« size. A separate wedge set (3 wedges) is offered for »L« and »XL« sizes of the Walker.

Before fixing the heel wedges assure an exact fit by placing them under the sole of the foot after having assembled them in layers in their correct order. Once customized fix the largest wedge to be used to the heel area of the sole of the Walker by peeling off the protective film of the adhesive tape first. Then fix the two following layers above accordingly.

1. The Walker should be worn permanently for 2 weeks including the 3 corresponding heel wedges (plantar flexion of the foot at 22°). It may only be removed for personal hygiene. As long as the Walker is off, the foot must be kept in plantar flexion. The foot may be flexed as far as it does not cause pain.
2. One heel wedge will be removed after 2 weeks, which usually is the upper one. For a better fit it is also possible to remove the lowest one (plantar flexion of the foot at 16°).
3. Another wedge will be removed after another 2 weeks (plantar flexion of the foot at 16°). The last wedge will be removed from the Walker 2 weeks later. In order to get used to the 90° position of the upper ankle joint it is recommended to continue wearing the Walker for several days.

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