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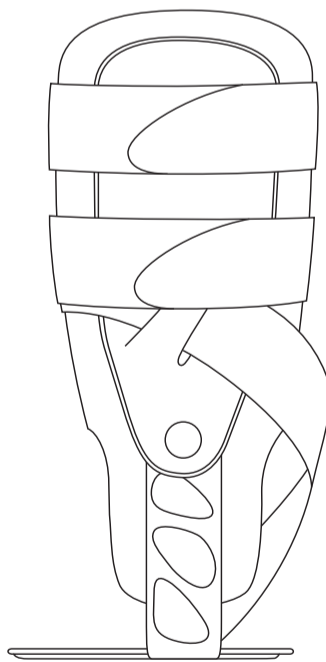
### Body Armor® Embrace

Ankle joint orthosis

- > The Embrace ankle joint orthosis can be customized individually to match the foot after a comparatively short time wearing it.
- > For maximum stability we recommend to wear athletic shoes.
- > For advanced comfort please wear an absorptive and highly cut sock.

#### Application of the Body Armor® Embrace in the following order:

1. Check whether your shoe has a removable insole.
2. a. If your shoe contains a removable insole or footbed, place the Body Armor® Embrace underneath the insole or footbed. In this case we recommend to remove the enclosed black foam pad for the heel and to not use it.  
b. If the insole of your shoe cannot be removed, take off the adhesive backing of the enclosed black foam pad for the heel and fix the pad centric to the upper surface of the bottom shell.
3. 2. Open the bungee closure system in order to simplify entering the orthosis.
3. Insert your Body Armor® Embrace into the shoe for the bandage to assume the following position as seen in image B (broad sole cupping underneath the heel and opening at the outside of the foot). Please take care that the ankle joint orthosis fits closely around your ankle and foot. Try to place the orthosis sole as far to the rear end of the shoe as possible.
4. Close the long lower velcro strap first (runs circular through the loop at the inside) by fixing it the lowest possible at the front of the splint. Now also fasten the upper velcro strap by laying it across the velcro spots at the inside.
5. If the velcro strap is too long, you may shorten it individually. Unfasten the double-catching end of the bungee closure system (y-form / with logo) and cut off the hook strap at the desired length. Afterwards fix the hook strap to the loop strap again.
6. Then tie your shoes as usual.

#### Using the orthosis for prevention or after an acute period of injury / Remodeling the Body Armor® Embrace:

If you wear the ankle joint orthosis for prevention or if you are able to resume your regular daily routine after an acute injury, you can remodel your Body Armor® Embrace as follows in order to achieve a more comfortable fit:

- > Remove the foam cushions from the lateral fixing points.
- > Shorten their length by cutting off the thin lower end at the reshaped edge (image A).
- > Reinsert the cushion after shortening and apply the orthosis as usual.

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