# Bungee Wrist Splint

**USA / GB Indication**

- > For injuries, mild sprains and strains
- > For tendonitis, carpal tunnel syndrome and tenosynovitis
- > For carpal tunnel, wrist drop and rheumatoid arthritis

**Contraindication**

- > Open fractures

**Technical data**

**Sizes:**  
The Bungee Wrist Splint is available in 5 sizes depending on the circumference of the wrist. The length for all sizes is 20 cm. The splint is available for the right and left hand individually.

Size	Wrist circumference
XS	13 – 16 cm
S	16 – 19 cm
M	19 – 21 cm
L	21 – 23 cm
XL	23 – 26 cm

**Colour:** Black

**Material:**  
Outer material: PVC  
Lining: Polyester fabric  
Elastic rubber bands: Isoprene  
Multi-layer cotton jersey: Cotton elastic fabric

**Maintenance:**  
The product does not require maintenance. The Bungee Wrist Splint is designed for the use of one person only.

**Cleaning instructions**

The Bungee Wrist Splint is to be washed in the enclosed laundry bag. It is recommended to remove the inner metal plate from the splint first and to close the velcro® straps. Please do not tumble dry, but air dry the splint.

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#### Bungee Wrist Splint Wrist Splint

The Bungee Wrist Splint is designed to immobilize and stabilize the wrist allowing for full function of the fingers. The metal plate, which is worked into the splint and extends to the palm area of the hand, enables a total immobilization of the wrist. The part of the hand subsequent to the palm area allows an almost unrestrictedly movement of the fingers. Padded inner lining, perforated velour outside and elastic rubber bands provide the Bungee Wrist Splint with a comfortable fit. A dorsally applied cotton liner prevents the rubber bands from directly touching the skin. Eight elastic rubber bands are bundled by a velcro® strap ensuring high compression, a perfect fit as well as a smooth and easy use of this wrist splint.

#### Recommendations for use:

The bungee closure system enables a quick and easy application of this wrist splint by oneself. For maximum stabilization and a comfortable fit we recommend putting on the splint as follows:

1. Loosen the velcro® strap and widen the splint.
2. Apply the splint by leading the thumb completely through the thumb loop.
3. Pull all of the elastic bands by means of the velcro® strap and wrap them around the splint. (fig. 1)
4. Test for a secure and comfortable fit of the splint and subsequently close the velcro® strap. (fig. 2)
5. If required tighten the rubber bands once more.
6. The splint is ready for use. (fig. 3 + 4)

#### Caution

This product is intended for rehabilitative purposes only. Please consult your physician or physiotherapist for sensible use of the Bungee Wrist Splint. If you experience any pain, swelling, sensation changes, or any unusual reaction while using this product, discontinue use and contact your physician immediately.

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