USA/GB To the patient

Your physician has dispensed to you a very unusual looking shoe; one engineered and designed to remove weight from the part of the foot that underwent surgery. This design permits the quickest possible healing of ulcerations, surgical procedures, fractures, etc. while still allowing you to maintain your mobility. The shoe (OrthoWedge, HeelWedge®, MedSurg™/Original) is more difficult to walk in than conventional footwear so please read the following precautions.

Caution

- > Do not attempt to drive while wearing the shoe.
- > Use extreme caution when climbing stairs, stepping up onto curbs and walking on uneven surfaces.
- > Walk more slowly than usual. You are not wearing a normal street shoe and more clearance is required for the wedge sole.
- > Excessive weight (obese patients) may cause a breakdown of the shoe's functional sole. In case the sole must be fixed due to extensive wearing, please contact your physician or orthopedic technician.
- > Recommendation to wear bandages or socks.

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111027

Manufactured by:
DARCO Int. Trading (Shanghai) LTD
No. 128 Lane 3458 Hu Min Road
Shanghai 201108, China

DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Toll-free 800.999.8866
Phone 304.522.4883
Fax 304.522.0037
www.darcointernational.com

DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 (0) 88 07 92 28-0 | Fax -22
Info@darco.de, www.darco.de


EC REP

OrthoWedge HeelWedge® MedSurg™/Original


DARCO


OrthoWedge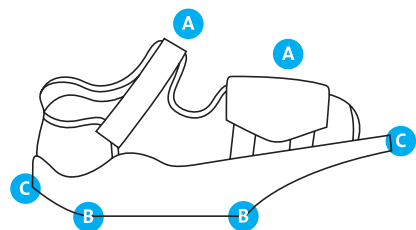

HeelWedge®

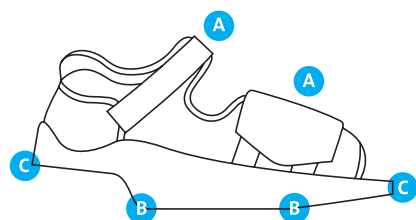MedSurg™/Original

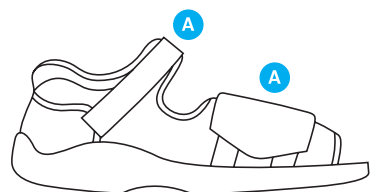USA/GB Instructions for use

OrthoWedge

Shoe to reduce pressure on the forefoot

HeelWedge®

Shoe to reduce pressure on the heel

MedSurg™/Original

Post-op shoe

1. Open the shoe (A).
2. Insert the foot into the opened shoe.
3. Make sure that the foot is positioned securely in the shoe.
4. Close the shoe with the hook and loop fasteners (A). So that the fit is snug, but not too tight.

For the **OrthoWedge** and **HeelWedge®**
(not applicable for MedSurg™/Original):

5. Only roll as far as the inner edges (B) when walking. Rolling as far as the outermost edges (C) can have a negative effect on the results of surgery.
6. When worn for longer periods, it is recommended that the height of the other foot is adapted to match. The shoe should not be worn for longer than 6 weeks.

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