Puzzle Insole®

USA/GB	Indication
	<p><b>Little toe module (slender)</b></p> <ul style="list-style-type: none"> <li>&gt; Tailor's Bunion (Digitus Superductus)</li> <li>&gt; Jones Fracture (Puzzle module ends at the Lisfranc's joint line)</li> </ul> <p><b>Great toe module (big)</b></p> <ul style="list-style-type: none"> <li>&gt; MTPI-Arthrodesis</li> <li>&gt; Austin-, Scarf- and Akin-Osteotomy</li> <li>&gt; Closing- and Opening-Wedge-Osteotomy</li> <li>&gt; Crescentic-Osteotomie</li> <li>&gt; Lapidus-Arthrodesis (TMT1-Arthrodesis)</li> <li>&gt; Youngswick-Osteotomy (Puzzle module ends at Cuneiforme I)</li> </ul> <p><b>Centerpiece</b></p> <ul style="list-style-type: none"> <li>&gt; Weil- and /or Helal-Osteotomie I – IV</li> <li>&gt; Morton's Neuroma</li> <li>&gt; Hammer toe Procedures</li> <li>&gt; PIP- / DIP-Arthrodesis</li> <li>&gt; Metatarsal Fractures II – IV and PIP Arthrodesis</li> <li>&gt; Shortening-Osteotomies Metatarsals II – IV (Puzzle module ends at the proximal end of Metatarsals II – IV)</li> </ul>

**Contraindication**

> Open fractures

**Technical data**

> compatible with MedSurg™

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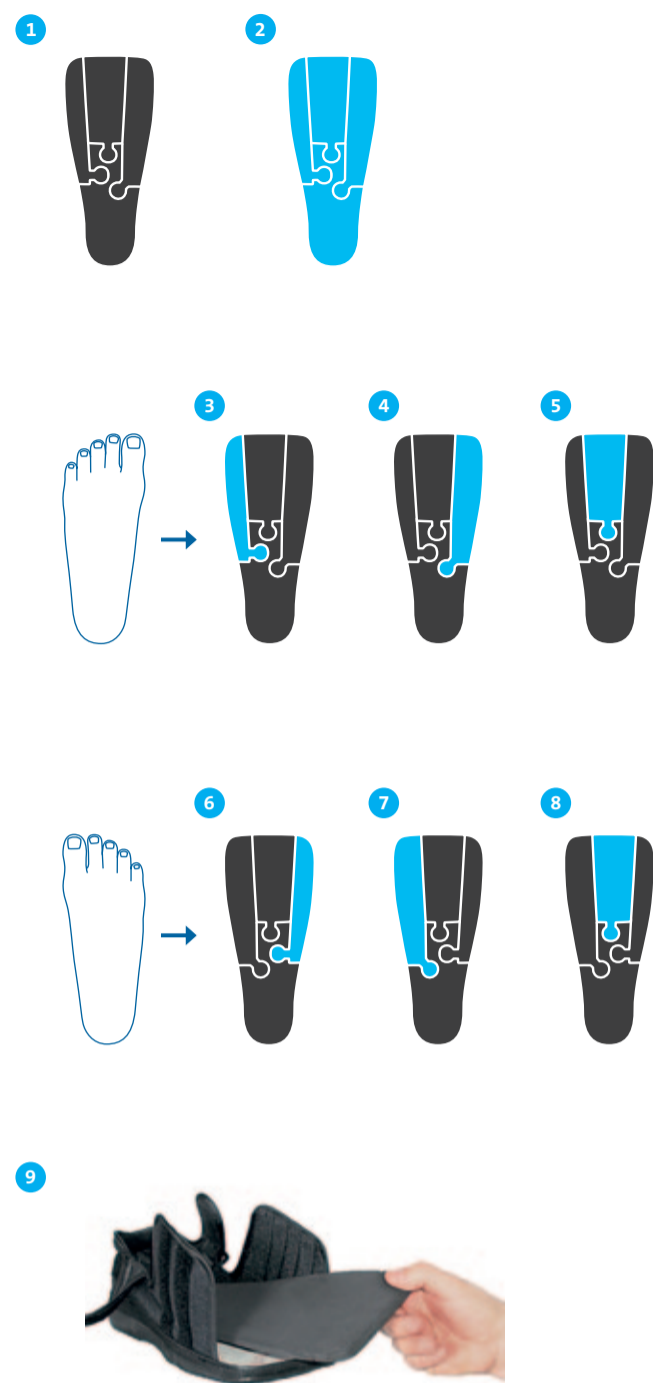
**Puzzle Insole®**  
Modular Insole

The modular insole was developed for plantar pressure redistribution with specific use after forefoot- and midfoot-operations. By simply replacing the interchangeable modules, the patient can achieve the perfect pressure distribution for each indication. Additional pressure relief of up to 22% in the front area is effected by the wedge-shaped sole that tapers toward the heel.

**Recommendations for use:**

1. The black sole (fig. 1) serves as carrier model. The modules of the blue sole (less shore-stiffness, fig. 2) are used for multi-indication pressure distribution.
2. Insert a blue module (slender, big and /or medium) into the black carrier sole where the effect of pressure relief is focused on.

**Possible combinations for the left foot:**

- > Little toe module (slender), (fig. 3)
- > Great toe module (big), (fig. 4)
- > Centerpiece (fig. 5)

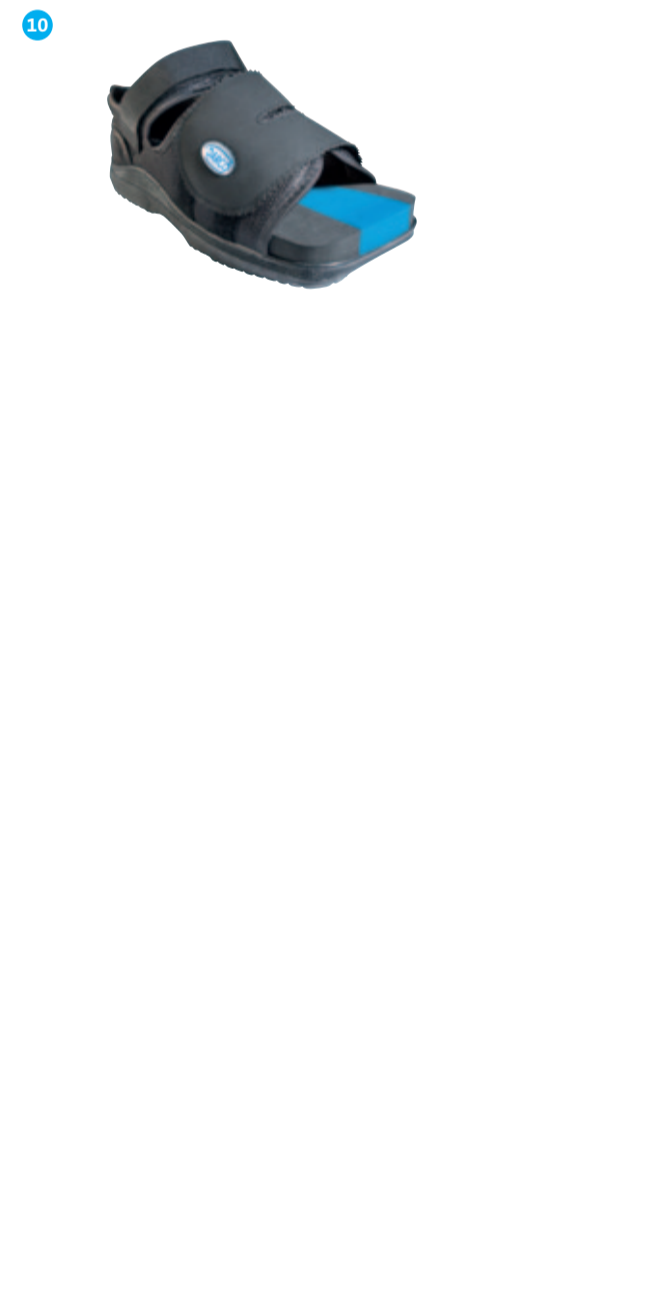
**Possible combinations for the right foot:**

- > Little toe module (slender), (fig. 6)
- > Great toe module (big), (fig. 7)
- > Centerpiece (fig. 8)

3. Remove the original sole from the shoe. (fig. 9)
4. Reinsert the adjusted Puzzle Insole® into the shoe. (fig. 10)  
Ready to go.

**Caution**

- > The modular insole fits the right and left foot equally. Please ensure correct application!
- > The sole has to always include all parts (4 parts) for achieving selective off-loading. Do not use if one module is missing.
- > It is recommended to wear bandages or socks.

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