## To the patient

Your physician has dispensed to you a very unusual looking shoe; one designed and engineered specifically to guarantee that the height of the TwinShoe exactly matches that of its counterpart on the treated foot.

The design of the TwinShoes provides sure-footed security while standing and a comfortable, natural toe-off movement when walking. Because walking becomes more complicated with the combination of TwinShoe and e.g. OrthoWedge™ than with conventional footwear, please read all instructions carefully before use.

The TwinShoe is a height-matched counterpart for the following products: OrthoWedge™ /-light, HeelWedge™ and WCS™. In the case of bad weather, a toe cap for the TwinShoe is also available.

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
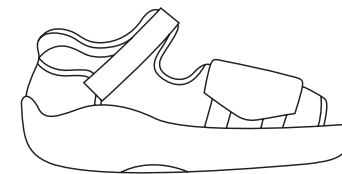
## TwinShoe
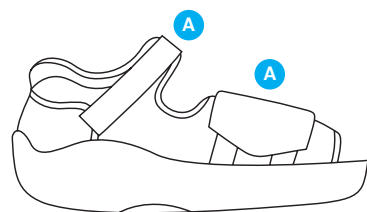
### TwinShoe (matching shoe)

1. Open the shoe (A).
2. Insert the foot into the opened shoe.
3. Make sure that the foot is positioned securely in the shoe.
4. Close the shoe with the hook and loop fasteners (A). So that the fit is snug, but not too tight.

### Caution

- > Do not attempt to drive while wearing the shoe.
- > Use extreme caution when climbing stairs, stepping up onto curbs and when walking on uneven surfaces.
- > Walk more slowly than usual. You are not wearing a normal street shoe and more clearance is required for the wedge sole.
- > The functional design of the shoe affects the stability of the sole, which might cause a breakdown in case of false utilization.

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