USA/GB Indication

- > For treatment of diabetic foot syndrome
- > For healing of open wounds and pressure ulcers and ulcerations
- > For pressure offloading
- > Postoperative after toe and forefoot surgery

Contraindication

- > Open fractures

Technical data

Size:
The shoe is available in seven sizes for the right and left foot.

Material:
Outsole: PU
Upper material: Natural leather

Maintenance:
The product does not require maintenance. The WCS™ is designed for the use of one person / patient only.

Plantar layer system:			
Material	Density	Thickness	Durometer*
	EVA	soft	6,3 mm 26
	EVA	medium	6,3 mm 42
	EVA	firm	6,3 mm 50
	Insole foam	high	3,1 mm 29

*The insole durometers may vary within 2° to 3° of those listed.

Cleaning instructions

When slightly dirty clean the WCS™ shoes with a damp cloth or a soft brush. Please do not use strong solvents for cleaning (alcohol etc.). Do not tumble dry or on the stove. Allow to air dry. The enclosed insoles can be washed with warm water and soap or cleaned with regular disinfectants.

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Manufactured for:
DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA

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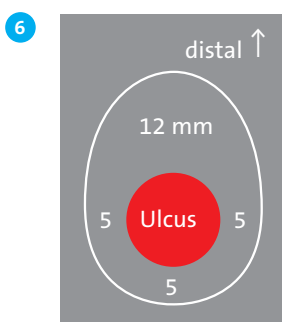
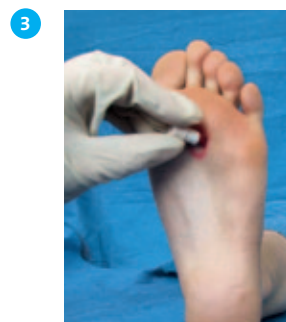
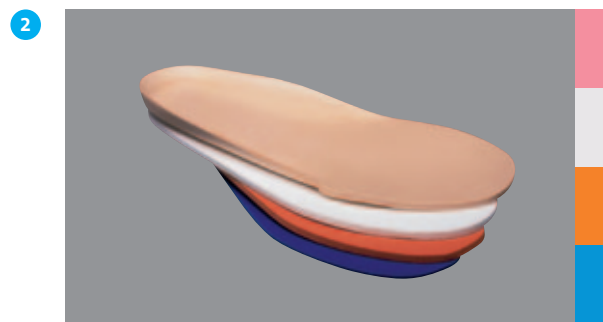
WCS™ Wound Care Shoe System

Closed wound care shoe

The WCS™ Wound Care Shoe is available individually for the right and left foot. Determining the correct shoe size is important for the perfect functionality of the shoe. On the inside of the sole you will find a »D« which should be placed distally just in front of the first metatarsal head. This will ensure that the pressure point of the rocker sole will be located distally to the metatarsal head. The velcro® closure system enables to easily and comfortably open and close the wound care shoes. The WCS™ system in particular supports patients limited in their mobility. For a proper stabilization of the foot the velcro® straps should be closed accordingly. The proven removable insole system with its 4 layers (fig. 2) is already included in the shoe (fig. 1) or enclosed with delivery and can be individually adjusted (fig. 5).

Marking the ulceration and customizing the insoles:

1. Mark the pressure ulcer with the enclosed marking pen onto the foot of the patient (fig. 3)
2. The patient should now stand up in the shoe/s onto the insoles which then pictures the ulcer onto the upper layer of the insole.
3. Puncture the two layers following the upper layer by penetrating this specific layer with a sharp tool. Then mark the area to be cut out (fig. 4).
4. Draw the areas to be removed onto the lower side of the layers as described in fig. 6.

Recommendations for use:

- > The Wound Care Shoe is mainly used for the treatment of open and closed ulcerations and ulcers (Malum perforans). The period of application depends on the grade of ulceration.
- > Daily wear with indoor and outdoor use.

Insole cutting:

- A No plantar open ulcers
B Level of ulceration 0 and 1, Wagner Scale, open wounds, bevel cut with approx. 30°
C After wound healing (of B)
D Level of ulceration 2 and up, Wagner Scale, open wound
E After wound healing (of D)
F Cross section depicting offloading lesion under third metatarsal head

Caution

This is a medical device for care treatment. For using the WCS™ system we recommend to consult your physician or orthopedic technician. If you experience pain, abnormal swelling, noticeable changes or other unusual reactions, discontinue use and contact your physician or orthopedic technician immediately. Recommendation to wear bandages or socks.

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