WCS™ Light
Wound Care Shoe System

USA/GB Indication

- > For treatment of diabetic foot syndrome
- > For healing of open wounds and minor ulceration
- > For pressure offloading
- > Postoperative after toe and forefoot surgery

Contraindication

- > Open fractures

Technical data

Size:
The WCS™ Light is available pairwise in seven sizes.

Material:
Outsole: PU
Upper material: Spandex / Polyester

Maintenance:
The product does not require maintenance. The WCS™ Light is designed for the use of one person / patient only.

Plantar layer system:

Material	Density	Thickness	Durometer*
EVA	soft	6,3 mm	26
EVA	medium	6,3 mm	42
EVA	firm	6,3 mm	50
Insole foam	high	3,1 mm	29

*The insole durometers may vary within 2° to 3° of those listed.

Cleaning instructions

When slightly dirty clean the WCS™ Light shoes with a damp cloth or a soft brush. Please do not use strong solvents for cleaning (alcohol etc.). Do not tumble dry or on the stove. Allow to air dry. The enclosed insoles can be washed with warm water and soap or cleaned with regular disinfectants.

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WCS™ Light Wound Care Shoe System
Closed wound care shoe

The wound care shoes are used in pairs. Determining the correct shoe size is important for the perfect functionality of the shoe. The velcro® closure system enables to easily and comfortably open and close the wound care shoes. The WCS™ Light system in particular supports patients limited in their mobility. For a proper stabilization of the foot the velcro® straps should be closed accordingly. The removable insoles are already included within the shoe (fig. 1). The proven four-layer-insoles are part of the shoe system (fig. 2) and can be customized individually (fig. 5).

Marking the ulceration and customizing the insoles:

1. Mark the pressure ulcer with the enclosed marking pen onto the foot of the patient (fig. 3)
2. The patient should now stand up in the shoe/s onto the insoles which then pictures the ulcer onto the upper layer of the insole.
3. Puncture the two layers following the upper layer by penetrating this specific layer with a sharp tool. Then mark the area to be cut out (fig. 4)
4. Draw the areas to be removed onto the lower side of the layers as described in fig. 6.

Recommendations for use:

- > The light wound care shoe is specifically used for minor ulcers. The period of application depends on the grade of ulceration.
- > Daily wear with indoor and outdoor use.

Insole cutting:

- A No plantar open ulcers
- B Level of ulceration 0 and 1, Wagner Scale, open wounds, bevel cut with approx. 30°
- C After wound healing (of B)
- D Level of ulceration 2 and up, Wagner Scale, open wound
- E After wound healing (of D)
- F Cross section depicting offloading lesion under third metatarsal head

Caution

This is a medical device for care treatment. For using the WCS™ Light system we recommend to consult your physician or orthopedic technician. If you experience pain, abnormal swelling, noticeable changes or other unusual reactions, discontinue use and contact your physician or orthopedic technician immediately. Recommendation to wear bandages or socks.

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