

DARCO Posterior Splint Fix & DARCO Posterior Splint Plus Fix Foot-Positioning Orthoses

Innovative orthoses for immobilising in a defined position



DARCO Posterior Splint Plus Fix



DARCO Posterior Splint Fix

Indications

For general use in conservative and postoperative therapy of lower leg, ankle and foot injuries:

- > Achilles tendon rupture
- > Pre- and postoperative support
- > Post-traumatic stabilisation / Immobilisation
- > Conservative support
- > Immobilisation following an accident
- > Short-term stabilisation (ligament rupture, luxation, fractures, wounds, strains, sprains, etc.)



DARCO

Direct postoperative use in the operating theatre means that tedious plastering or warming and shaping of a splint are no longer required. **This significantly shortens costly operating times!**



Positioning of the foot in the orthosis.



Fitting and circular closing of the webbing straps around the calf area of the orthosis.



Circular fitting of the narrow webbing strap around the forefoot.



The strap material adheres to the hook-and-loop straps on the outside of the plastic shell.

Fitting options for the straps



Fitting the three straps in a circularly around the calf area and the forefoot.



The narrow straps can be joined together with the hook-and-loop fastener. The long strap is then closed at the forefoot and diagonally at the ankle.

Features

- > For Achilles tendon pathologies in 20° club foot position
- > Optimal positioning in defined position with:
 - Three flexibly positionable elastic webbing straps (included in the device packaging) or
 - Conventional gauze bandages with integrated hook and loop strips on the outside

- > Soft and wipeable inner lining (PE foam)
- > Anatomically optimized fit
 - To avoid pressure points
- > Inherently stable
- > Universal fit for right and left



DARCO Posterior Splint Plus Fix Features

Version with moderate angular position (90° Plus)

- > Two oval cut-outs ensure good air circulation
- > Anatomically optimized fit + large-volume heel shape to avoid pressure points



DARCO Posterior Splint Plus Fix Foot-Positioning Orthosis

90°
Plus

SIZE	FOOT LENGTH OF THE SPLINT	CALF LENGTH OF THE SPLINT	ART. NO.	PZN
------	---------------------------	---------------------------	----------	-----

Colour: White-Blue

S	26.5 cm	34.0 cm	PSF1-90-P	17916909
M	29.5 cm	37.0 cm	PSF2-90-P	17916915
L	32.5 cm	40.0 cm	PSF3-90-P	17916921



Including a special absorbent fleece against wound secretion after the first post-op days



DARCO Posterior Splint Fix Foot-Positioning Orthosis

110°

SIZE	FOOT LENGTH OF THE SPLINT	CALF LENGTH OF THE SPLINT	ART. NO.	PZN
------	---------------------------	---------------------------	----------	-----

Colour: White-Blue

S	26.5 cm	34.0 cm	PSF1-110	17916938
M	29.5 cm	37.0 cm	PSF2-110	17916944
L	32.5 cm	40.0 cm	PSF3-110	17916973



Including a special absorbent fleece against wound secretion after the first post-op days