

WCS®

Wound Care Shoe

The complete solution for immediate care and targeted pressure relief for diabetics



- > For treatment of diabetic foot syndrome
- > For healing of open wounds, ulcers and ulcerations
- > For pressure relief
- > For use in the postoperative phase after toe and/or forefoot surgery



DARCO

Benefits at a glance

Inner lining

- > Seamless and ultrasoft
- > Disinfectable

Hook and loop fastener

- > Optimal fit
- > Ideal for patients with reduced mobility



Comfort

- > Inherently stable and durable
- > Wide opening
- > Accommodates bulky dressings

Heel cap

- > Optimal hold
- > Stable

Upper material

- > Can be adjusted individually
- > For targeted pressure relief

Outsole

- > Rocker-style sole
- > For indoor and outdoor use
- > No bouncing

Cover sole


- > Ultra-soft closed-cell Plastazote®

Marking

- > For precise size determination

Sole system

- > 4-sole system
- > Individually adjustable

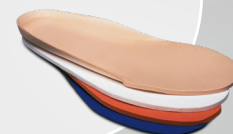

Optionally available:



Toe cap

- > Prevents foreign matter from entering the shoe
- > Protects against the cold and moisture
- > Elastic and breathable

Highest level of customization

The WCS® can be individually customized to every patient's foot in a very quick and easy process for targeted pressure relief.

Construction of the upper material

For targeted pressure relief in the dorso-medial or lateral part of the foot (Malum perforans), only the outer layer of leather of the shoe upper needs to be cut away in the area of the wound. The soft Plastazote® material of the inner shoe remains intact.



Construction of the insole

The exchangeable 4-sole system with a variety of Shore hardnesses ensures optimal pressure relief. Thanks to its design and the manual customization of the inner sole, precise plantar, medial and lateral pressure relief is provided.



SOLE STRUCTURE

A variety of Shore hardnesses for optimal pressure redistribution

Material	Density	Thickness	Durometer value*
Plastazote®	soft	6,3 mm	6*
EVA	medium	6,3 mm	42*
EVA	firm	6,3 mm	50*
Insole foam	high	3,1 mm	29*

* The values may vary within 2° to 3° of those listed.

Example of how to cut into the sole



Article List

SIZE ART. NO. SIZE ART. NO.

WCS®

Colour: Black

XS 35.0 – 36.0
 S 37.0 – 38.0
 S/M 39.0 – 40.0
 M 41.0 – 42.0
 L 43.0 – 44.0
 XL 45.0 – 46.0
 XXL 47.0 – 48.0

Right

WCS0B/R
 WCS1B/R
 WCS7B/R
 WCS2B/R
 WCS3B/R
 WCS4B/R
 WCS5B/R

Left

WCS0B/L
 WCS1B/L
 WCS7B/L
 WCS2B/L
 WCS3B/L
 WCS4B/L
 WCS5B/L

Toe cap

Right

TCWCS0/R
 TCWCS1/R
 TCWCS7/R
 TCWCS2/R
 TCWCS3/R
 TCWCS4/R
 TCWCS5/R

Left

TCWCS0/L
 TCWCS1/L
 TCWCS7/L
 TCWCS2/L
 TCWCS3/L
 TCWCS4/L
 TCWCS5/L




WCS®


> Sold in pairs

Sizes and Measurements


All measurements are defined in cm and correspond to the insole.
 DARCO shoe sizes represent a general reference only.
 The choice of shoe size is significantly influenced by the volume
 of the bandages and size of cast.

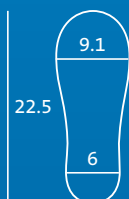

XS 35–36

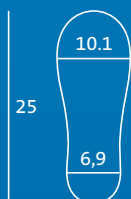


S 37–38

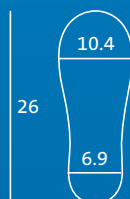


S/M 39–40




M 41–42

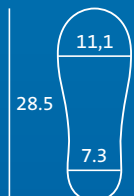

L 43–44

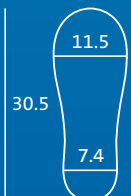

XL 45–46

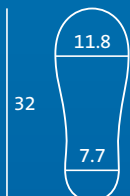

XXL 47–48



Product video

Scan the code and watch the product video.

You can find all DARCO product videos on our website: www.darco-europe.com or on our YouTube channel.