Relief Dual®

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NEW: self-closing
hook and loop fastener

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NEW CLOSING SYSTEM:

- > easy release
- > fast handling
- > individual shortening
- > slim design

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The new improved closing system of the Relief Dual® off-loading shoe ensures a firm and reliable connection. The closure is self-closing. It can be interlocked in any position simply by pressing the two sides together firmly.

Instructions for use

1. Open the shoe by means of the hook and loop straps (A, B).
2. Please ensure that the foot is properly positioned in the shoe.
3. Guide the upper hook and loop strap A through both D-rings on the shoe (picture 1).
4. The inner shoe tabs (under the hook and loop straps) can be optionally placed over each other (C).
5. Fold the strap on the outer side (left or right) around the tab and secure it firmly (picture 2).
6. Close the shoe with the other end of the strap by placing it on top (picture 3).
7. If needed, the length of the hook and loop strap A can be customized (with a scissor) to fit (picture 4).
8. Close the shoe with the lower hook and loop fastener B such that the DARCO logo is on the outside of the shoe.

Note: The shoe can be customized for the right or left foot by individually adjusting the hook and loop straps. The straps (A, B) are always closed on the outside of the foot.

Herausgeber | Issued by

DARCO (Europe) GmbH

Gewerbegebiet 18 | 82399 Raisting | Germany

Tel. +49 (0) 88 07.92 28-0 | Fax -22

info@darco.de | www.darco.de

info@darco-europe.com | www.darco-europe.com