# DARCO

## + Toe Plate (Zehenplatte) Night Splint Body Armor®

EC REP

DARCO (Europe) GmbH  
Gewerbegebiet 18  
82399 Raisting, Germany  
Telefon +49 (0) 88 07 92 28-0  
Fax +49 (0) 88 07 92 28-22  
info@darco.de | www.darco.de

180408

DARCO International, Inc.  
810 Memorial Blvd.  
Huntington, WV 25701, USA  
Toll-free 800.999.8866  
Phone 304.522.4883  
Fax 304.522.0037  
www.darcointernational.com

<b>USA/ GB</b>	<p><b>Technical data</b></p>
<b>Size:</b> Small (34 – 36) Standard (37 – 47) Can be worn on the right or left.	
<b>Colour:</b> Shell backing splint: Blue-White Soft pads: Grey-Blue Toe plate: Grey-Blue	
<b>Material:</b> Shell backing splint: Nylon Soft pads: Polyester Spandex Elastic ring: PVC Hook and loop fastener straps: Nylon Toe loop: Polyester Spandex Toe plate: Polyester Spandex	
<b>Maintenance:</b> The product does not require maintenance.	
<b>Construction features:</b> The Body Armor® Night Splint backing splint maintains the foot and lower leg at a 90° position. The elastic ring connector allows for off-loading the instep of the foot. The loops are made of breathable, elastic nylon /polyester fabric.	

### Indications

- > shortening of the plantar fascia
  - > heel spurs
  - > achilles tendonitis and tendonosis
  - > metatarsalgia
  - > talipes equinus and ankle contracture
- Contraindication:**
- > Plantar flexion contracture greater than 10°
  - > uncontrollable severe plantar flexion spasticity
  - > This product is not intended for extended periods of weight bearing or ambulation.

- > In case of an extremely upright Hallux Valgus position we recommend to use the Toe plate instead of the toe loop.

### Cleaning instructions

- > Open all the hook and loop fastener straps.
- > The shell and padding may be cleaned by wiping with a damp cloth.
- > The Toe loop/ Zeh plate is to be washed by hand with a mild detergent. The grey fabric for fixing the splint can be washed with 30°. This product is to be air dried and may not be tumbled dry or dried on the heater.
- > For cleaning the product you may use any commercially available sanitizer.

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#### Toe plate:

The Toe plate is an addition to the Body Armor® Night Splint splinting system. It provides the elevation of the five toes. This enables an almost complete even stretching of the flexor group.

The backing splint / toe plate fits right and left equally.

- Slide foot into the night splint, seating the heel snugly in the heel opening.
- Tighten, as needed, both sides of the heel strap together.
- Tighten, as needed, both sides of the foot strap together.
- Thread the calf strap. Tighten, as needed, both sides of the calf strap together.

#### Application of the Toe loop:

- Insert the big toe in the toe loop. The top of the toe should be level with the end of the toe loop.
- Using the elastic ring, place the big toe in a neutral or slightly elevated position\* and attach the ring to one of the hooks.
- Secure the toe loop tab to the bottom of the night splint.

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\*As treatment continues, the toe(s) may be elevated as comfort allows to permit a greater stretch to the plantar fascia. If you experienced any numbness or discomfort in the big toe(s) area, reduce the tension on the toe strap / plate in order to get an optimal and neutral position.

The backing splint / toe plate is designed for the use of one person only and not meant for multiple use.

## Caution

The Body Armor® Night Splint is a backing splint and not intended for walking. Please always follow these instructions for use. If side effects occur (e.g. skin swelling, pressure points, circulatory disorders), please inform your physician. If the product is handled incorrectly, its function and optimum protection can no longer be guaranteed. DARCO (Europe) GmbH does not assume any responsibility. This aid is a single-use product.

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