open  
®  
сnpoшmo

DARCO (Europe) GmbH  
Gewerbegebiet 18  
82399 Raisting, Germany  
Telefon +49 (0) 88 07 92 28-0  
Fax +49 (0) 88 07 92 28-22  
info@darco.de | www.darco.de

Made in China

USA / GB Indications
<ul style="list-style-type: none"> <li>&gt; Lesions on the medial MTP I joint, on the lateral MTP V joint and the entire dorsal and plantar forefoot</li> <li>&gt; for postoperative care</li> <li>&gt; for wound care</li> <li>&gt; arthritis</li> </ul> <p><b>Contraindications:</b></p> <ul style="list-style-type: none"> <li>&gt; open fractures</li> </ul>

Technical data
<p><b>5 sizes:</b> XS (33,5–37), S (37,5–39,5), M (40–41,5), L (42–44), XL (44,5–47)</p> <p><b>Colour:</b> Black</p> <p><b>Material:</b> Outsole: EVA Insole: EVA Upper material: Foam polyester mesh Inner lining: Brushed polyester Hook and loop fasteners: Nylon Hook and loop straps: Polyamide</p> <p><b>Maintenance:</b> The product does not require maintenance.</p>

Design features
<p>The upper material is made of breathable nylon mesh that can be stretched and adjusted lengthways and sideways. The outsole is made of shock-absorbing EVA and a joint pad that is positioned distally enabling easier roll-over. The base of the shoe is reinforced throughout. The option of completely opening the shoe, an adjustable closure, a variety of fastening options in the forefoot area and a removable sole allow individual adjustment.</p>

Cleaning instructions
<p><b>Hand wash:</b></p> <ul style="list-style-type: none"> <li>&gt; If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.</li> <li>&gt; Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.</li> </ul>

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


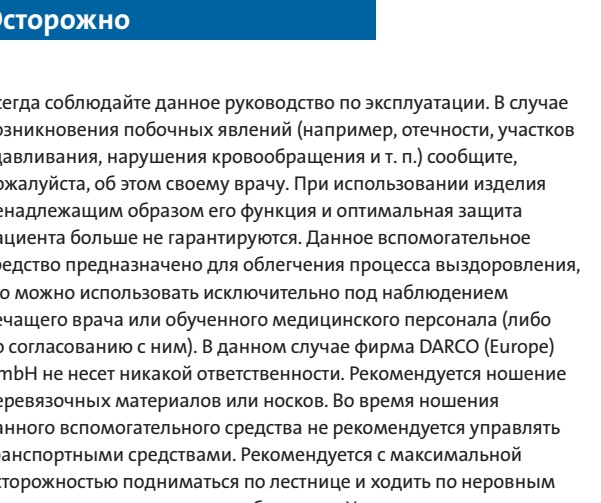
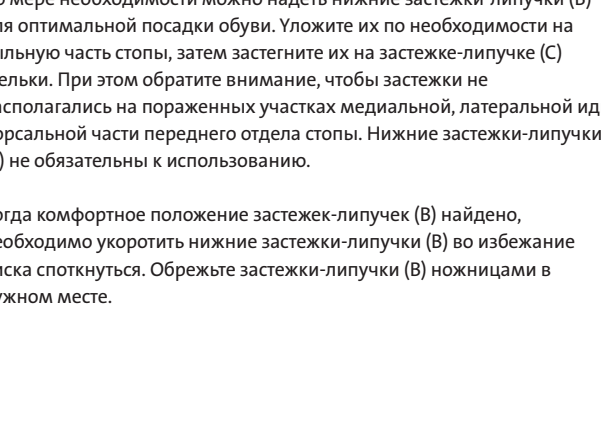
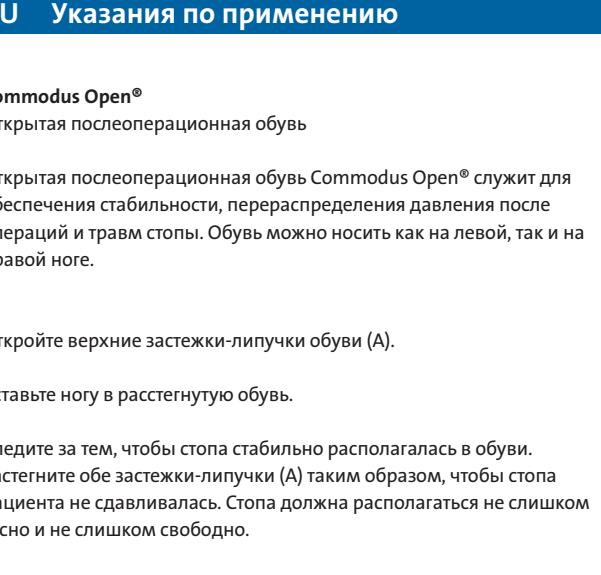
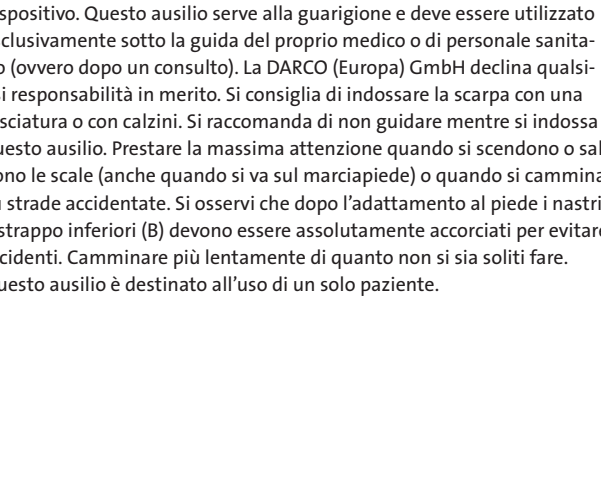
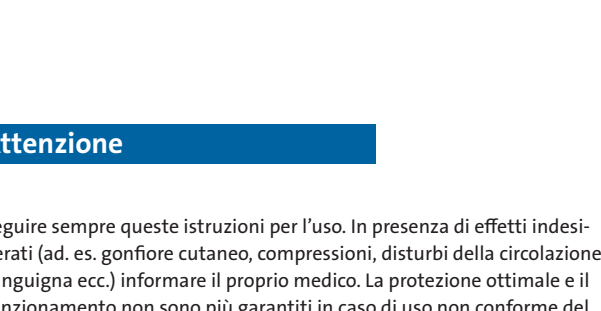
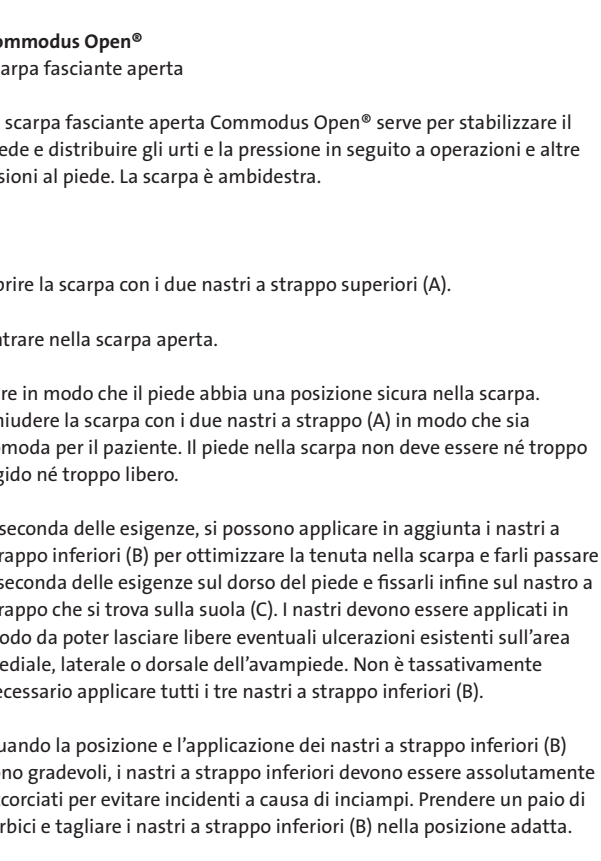
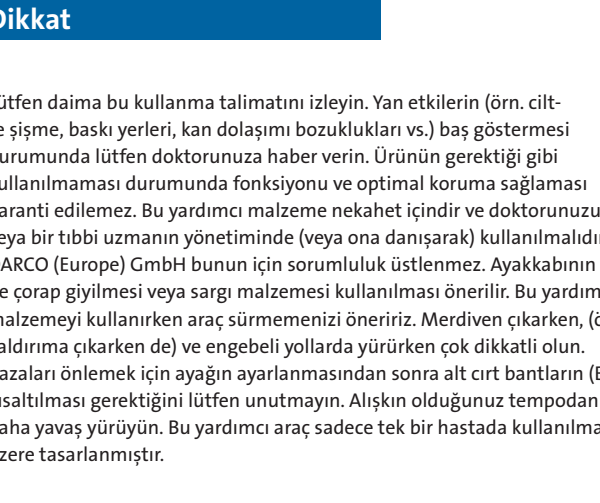
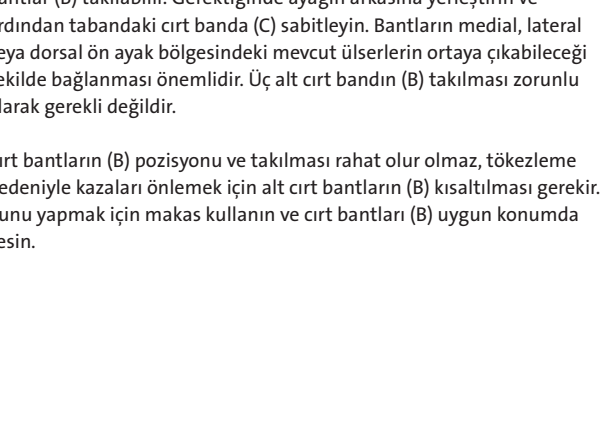
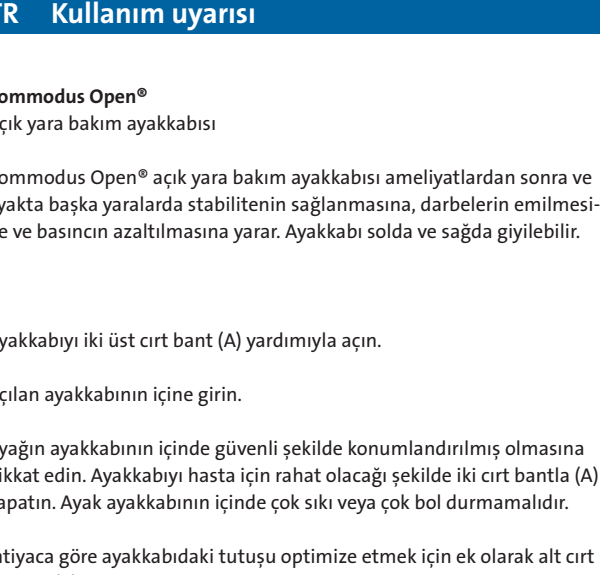
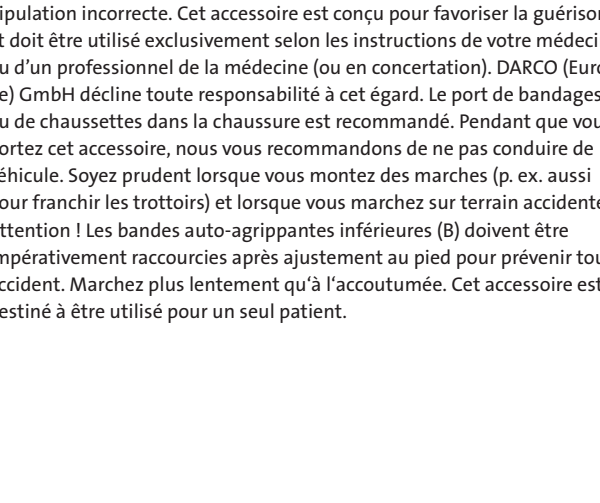
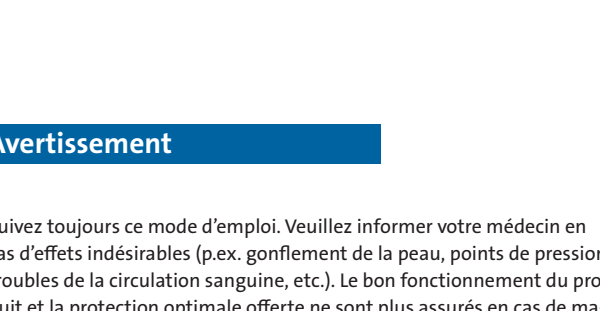
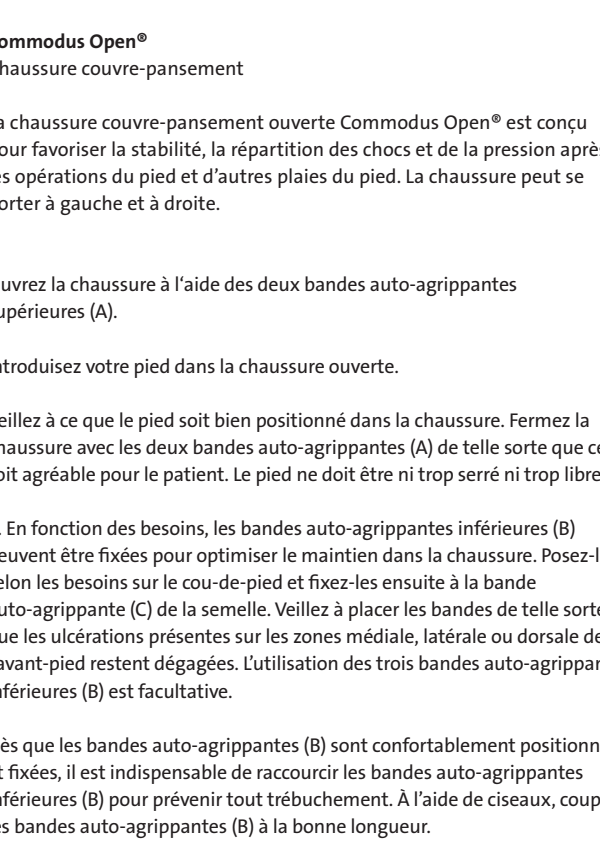
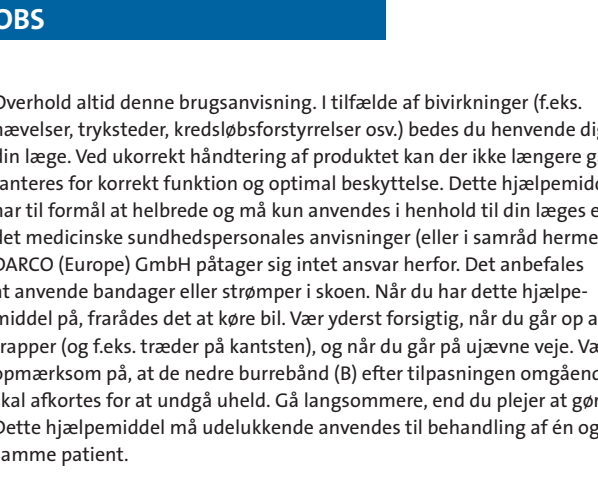
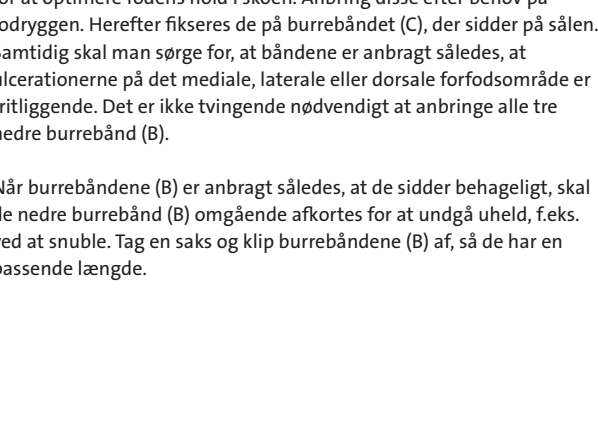
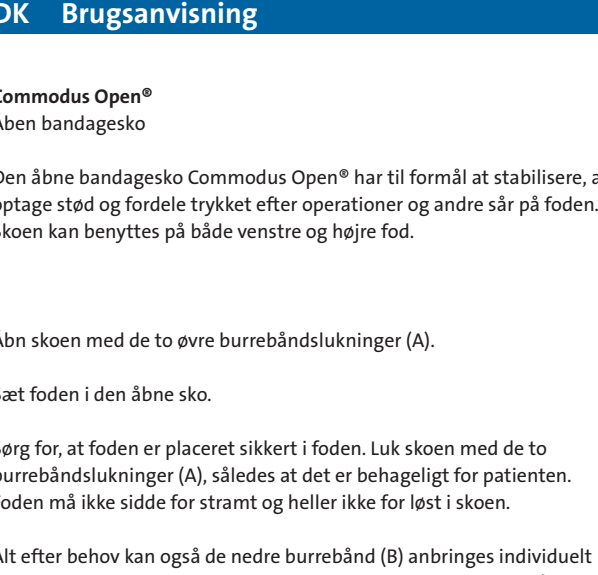
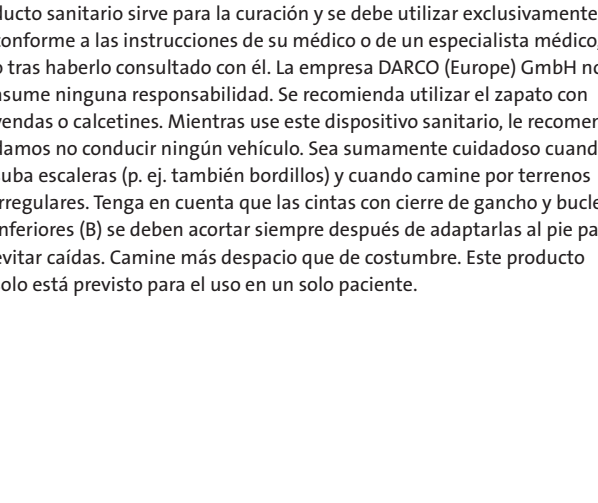
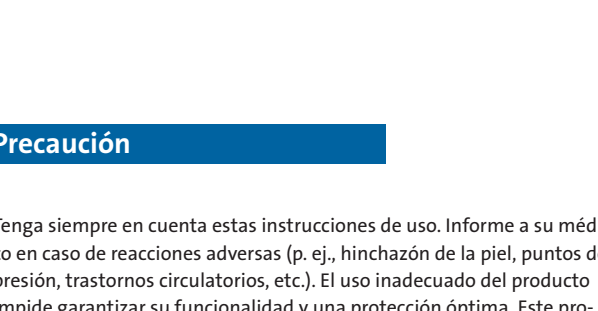
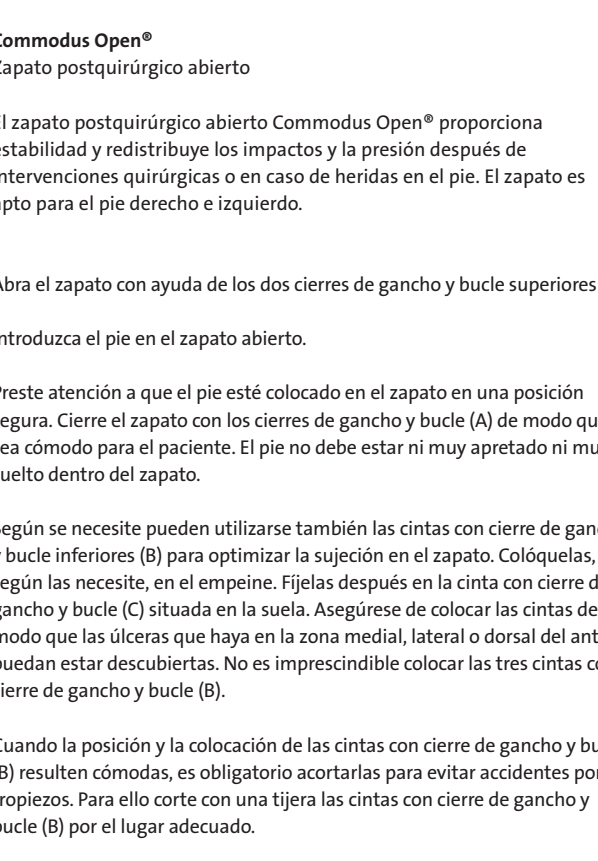
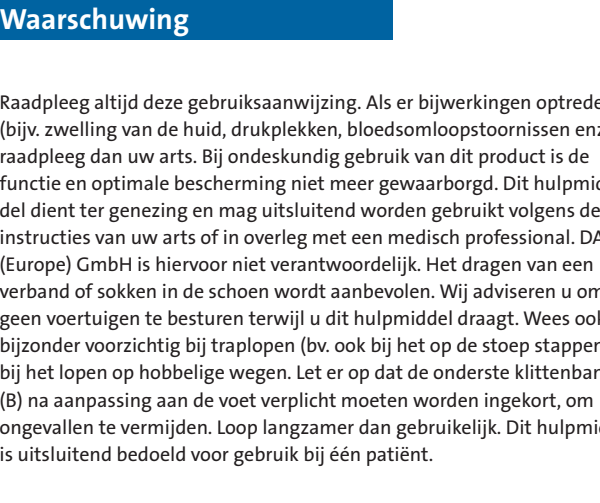
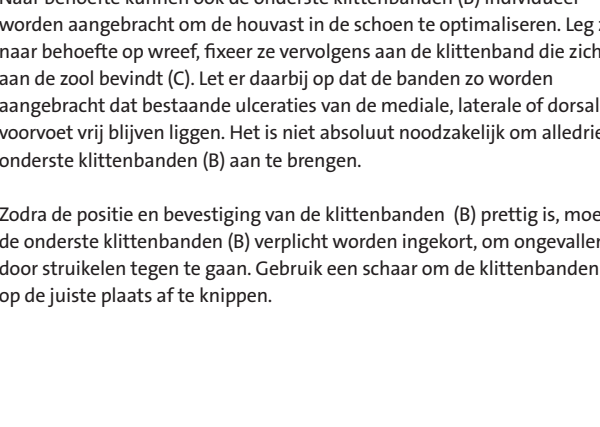
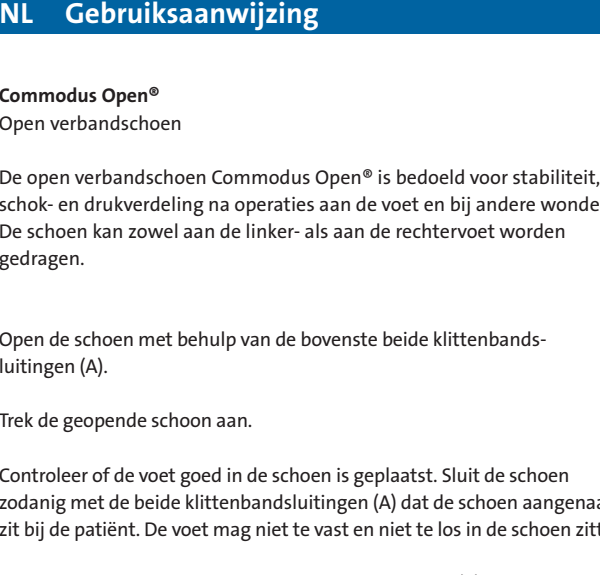
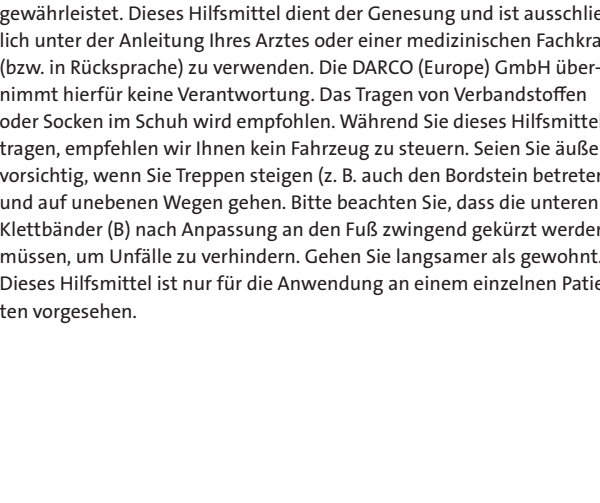
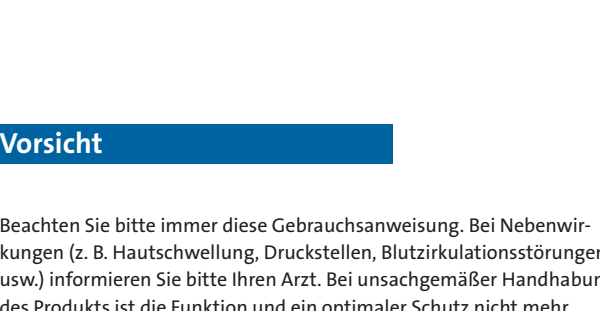
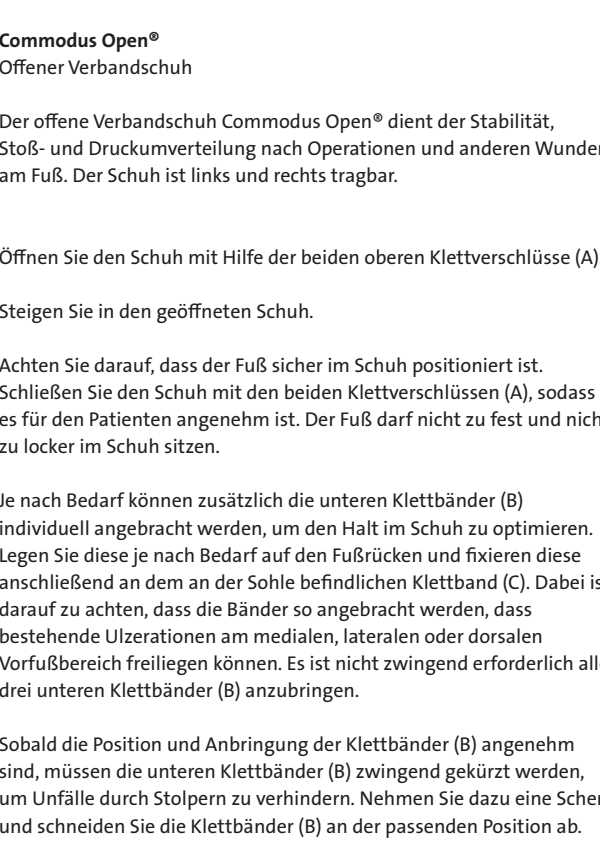
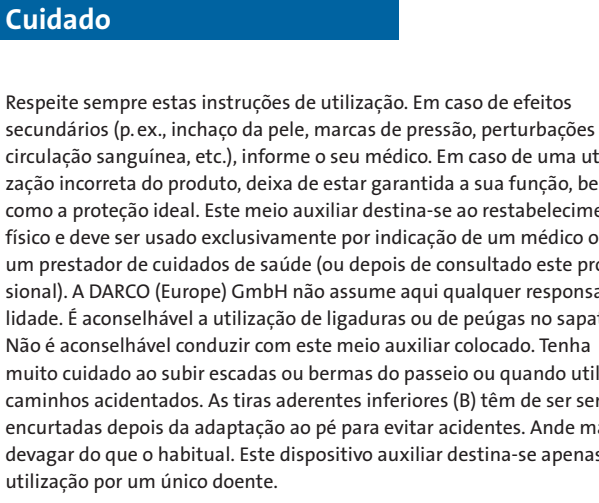
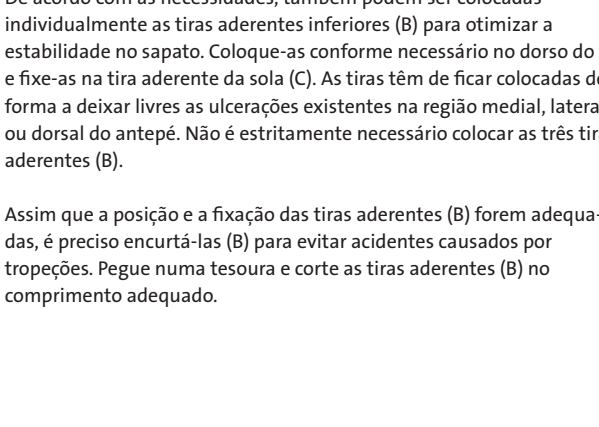
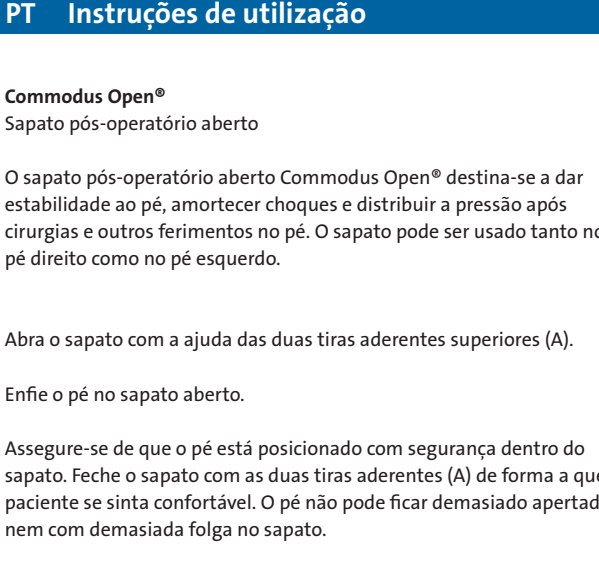
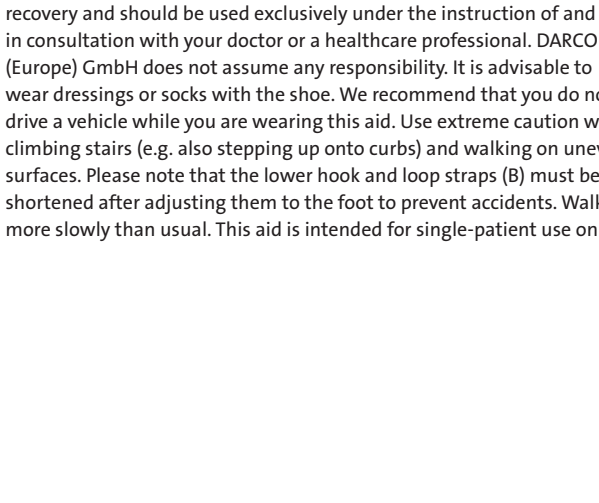
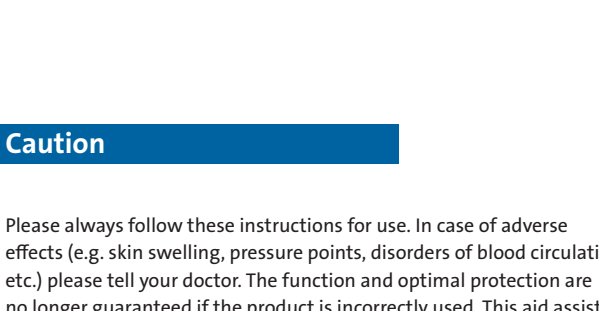
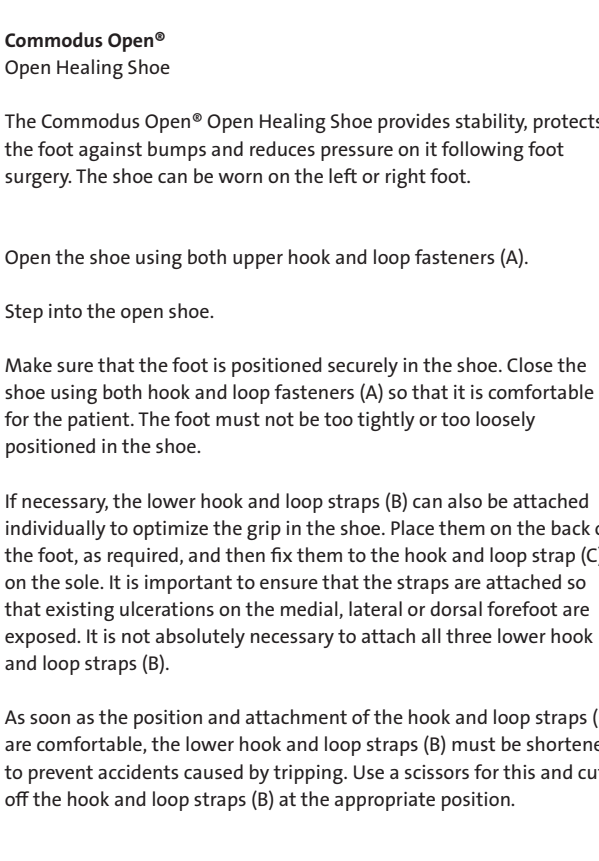
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
### Caution

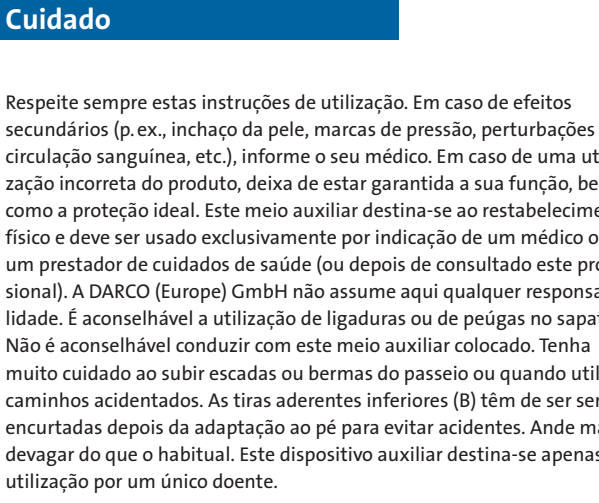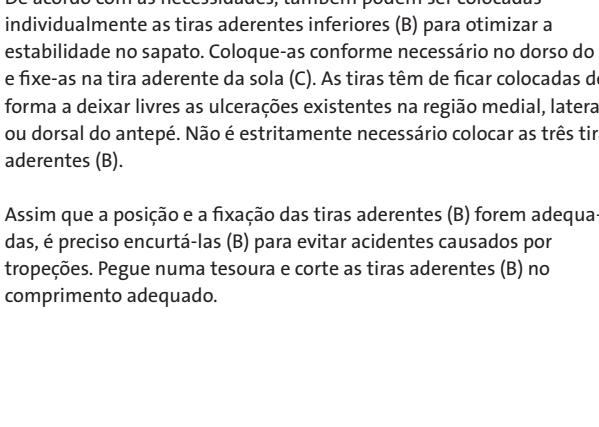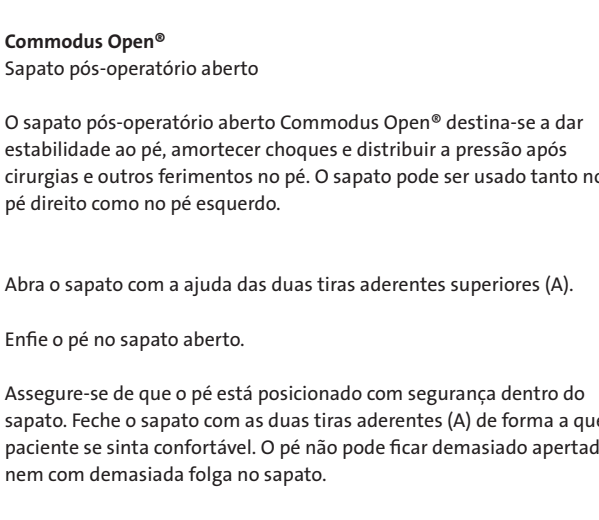Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. The function and optimal protection are no longer guaranteed if the product is incorrectly used. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings or socks with the shoe. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Please note that the lower hook and loop straps (B) must be shortened after adjusting them to the foot to prevent accidents. Walk more slowly than usual. This aid is intended for single-patient use only.

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