Coolaris®

Vertrieb / Distribution:

DARCO (Europe) GmbH

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82399 Raisting, Germany

Telefon +49 (0) 88 07.92 28-0 | Fax -22

info@darco.de, www.darco.de

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51674 Wiehl, Germany

USA/GB Technical data

Size:

circa 5 m × 8 cm

Material:

- > Backing material: gauze bandage made of 69% rayon, 31% polyamide
- > Cooling gel: hydrocolloid matrix containing natrium alginate and zinc-oxide

Caution

If the product is improperly treated, its function and optimum protection can no longer be ensured.

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Coolaris®

Elastic zinc-paste cooling bandage

Coolaris® combines by its improved formulation the good properties of a zinc-paste bandage with the ones of a cooling bandage. The circular cooling effect lasts for up to 5 hours and can be extended by spraying with water subsequently.

Instruction warning / risks

Coolaris® can also be applied directly to abrasions since the formulation is without any alcohol and oil. Once hardened Coolaris® supports everyday injuries and sports injuries like sprains, strains, contusions and dislocations. Coolaris® can be applied without pre-cooling. However, the storage in a fridge increases the cooling effect additionally.

- To be used as a semi-rigid support bandage for
 - > distension
 - > sprains
 - > dislocation
 - > strains
 - > deep venous thrombosis
 - > achillodynia
 - > contusion, bruises
 - > tendovaginitis
 - > phlebitis
 - > arthritides
 - > for reduction of post-traumatical edema in the joint area
 - > chronic venous insufficiency
 - > post-operative treatment
 - > 1st degree burns
 - > insect bite of any kind

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