DAW (DARCO Achilles Wedges)

- EN Heel Wedges for Walkers
- DE Fersenkeile für Walker
- ES Cuñas de talón para andador
- FR Cales de talon pour Walker
- IT Zeppe per Walker
- NL Hielwiggien voor walkers
- DA Hælkiler til Walker
- SV Hålkilar för Walker
- PL Klíny do ortezy typu Walker
- SL Klinasti poviški za peto
za pripomoček Walker
- HR Ulošci za petu za ortožu
za hodanje
- PT Cunhas de calcanhar
para Walker

DARCO

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists

recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example.

We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual.

This aid is intended for single-patient use only.

Purpose

Heel wedges for defined adjustable positioning of a lower leg/foot orthosis.

Indications / Contraindications

Indications

- > Achilles tendon ruptures
- > Postoperative use for Achilles tendon suture
- > Postoperative use for removal of Haglund's exostosis

Contraindications

- > Forefoot conditions
- > Plantar foot ulcers

Design features

DAW heel wedges consist of 3 wedges in different lengths. The wedges are placed on top of each other according to the length of the foot and the desired angle of plantar flexion and fixed to the insole in the heel area of the walker. The wedges come in one standard size. They can therefore be used universally with all sizes of walkers. They can be used on the right and left.

EN Use

1. Open the walker and leave the insole inside the walker.
2. Place the wedges in the heel area of the walker, corresponding to the degree of plantar flexion that is therapeutically required. The wedges are placed on the insole of the walker.
3. A wedge has a height of approx. 11 mm. Depending on the number of wedges used, the height varies and the angle of plantar flexion can, thus, be adjusted individually. 3 wedges are equivalent to approx. 120° slant.
4. Fix the wedges to the insole of the walker using the adhesive dots.
5. Close the walker.
6. Depending on the course of therapy, the wedges can be removed and, thus, the height adjusted.

Combination with other products

Relief Insert® Walker 2.0
Relief Insert® Walker 2.0 Achill

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling/cleaning.

Technical data

Size: Universal
Contents: 3 wedges
High wedge: Approx. 11 mm per wedge; equivalent to approx. 120° plantar flexion
Colour: Grey
Material: EVA

Maintenance

The product does not require maintenance.

Cleaning instructions

Hand wash

If the wedges are slightly soiled, they can be wiped with a damp cloth. Do not dry the wedges in a dryer or on a radiator. Leave the wedges to dry at room temperature.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. The product can be disposed of with household waste.

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