USA / GB Indications

- > for extension preoperative
- > postoperative and for prevention

Contraindications:
> open fractures

Caution

The heel padding should be removed every 6 – 8 hours to discover discoloring of the foot's skin or changing of the blood circulation in time. The Heelift® Traction Boot for extension is only designed for the use with bedbound patients. In case of improper handling, the function and maximum safety of the product can no longer be guaranteed. This aid is intended for single-patient use only.

Technical data

Size:
Universal: 25 – 38 cm calf circumference

Colour:
Beige

Material:
Suspension Pad: Latex free polyurethane foam
Hook and loop fastener: Nylon

Maintenance:
The product does not require maintenance.

Construction features:
Suspension padding with additional traction device with traction bar and traction rope.

Cleaning instructions

Cleaning: Remove traction device before cleaning.

- > **To hand wash:** Use mild cleanser and warm water.
- > **To machine wash and dry:** Secure straps and place in net laundry bag or pillowcase (60° C).
- > **To sanitize after washing:** Use an antiseptic spray.
- > **To sterilize:** Use standard protocols with a steam autoclave. Allow a few hours to cool and dry completely.

For cleaning you may use water, standard detergents and disinfectants. The straps should be always closed during the process of cleaning. When washed at high temperatures, always allow the Heelift® Traction Boot to cool and dry before re-opening.

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Heelift® Traction Boot

Preoperative – the Heelift® Traction Boot for extension helps to stabilize femur fractures prior to surgery and enables to apply straight skin traction with up to 4,5 kg (Buck's traction).

Postoperative – without traction device – the suspension boot helps the prevention and healing of open wounds in the heel area during rehabilitation.

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
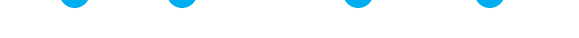
Application of the suspension boot

- preoperative for preparing the extension or
- postoperative for prevention

- Open up the Heelift® Traction Boot. Remove the spare pad. Please keep the spare pad available; it could be needed to further customize the boot later on.
- Place the patient's foot into the extension boot and ensure that the heel is positioned directly above the heel suspension opening. The heel should hang over the bottom elevation pad. (If legs are very long or short, you may take the elevation pad and move it forwards or backwards.)
- For closing pull the hook and loop fastener straps (C) over the leg towards the D-rings. Thread the straps through the D-rings. Use the hook and loop fastener closures to secure the straps. Leave foam flaps slightly open to provide additional ventilation.
- Test for the proper fit by trying to remove the closed extension boot. In case of free movement of the boot the spare pad may be applied in the toe area by removing the adhesive backings only from the inner side of the suspension padding. Your fingers should be able to fit comfortably between the heel opening and the bed. The boot should not be placed neither too loose nor too tight around the leg. The hook and loop fastener straps should not be in contact with the skin on top of the leg.
- In order to prevent lateral rotation of the foot use the extra pad and place it on the left or right side along with the tibia. Remove the adhesive backing and press the pad laterally onto the suspension padding.

Application of traction device

- Remove the long hook and loop fastener attached traction straps (D) from the sides of the extension boot.
- Unravel the traction rope (A) and double knot one end to the rope attachment bar (B).
- Fix the rope attachment bar with the hook and loop fastener traction straps to both sides of the Heelift® Traction Boot. Place the plate for the traction bar to be outside.
- Pass the unknotted end of the traction rope through the prepositioned traction pulley on the cross bar attached to the frame at the foot of the bed.
- Pull firmly on the rope and create a short 2,5 cm large loop in the rope with a double square knot (not a slip knot) just beyond the pulley.
- Attach a weight hanger with the prescribed weight (max. 4,5 kg).
- Elevate the extension boot together with the traction bar so that the Heelift® Traction Boot and the leg are 0,6 cm above the bed. You should be able to fit your finger between the heel opening and the bed.
- If the leg is externally rotated, apply the spare pad beneath the traction straps on the outside of the traction boot.
- To avoid heel cord contracture, secure the extra pad in a vertical position behind the sole of the foot.

USA/GB Instructions for use

Heelift® Traction Boot

Extension Boot

Preoperative

– the Heelift® Traction Boot for extension helps to stabilize femur fractures prior to surgery and enables to apply straight skin traction with up to 4,5 kg (Buck's traction).

Postoperative – without traction device – the suspension boot helps the prevention and healing of open wounds in the heel area during rehabilitation.

Application of the suspension boot

- preoperative for preparing the extension or
- postoperative for prevention

- Open up the Heelift® Traction Boot. Remove the spare pad. Please keep the spare pad available; it could be needed to further customize the boot later on.
- Place the patient's foot into the extension boot and ensure that the heel is positioned directly above the heel suspension opening. The heel should hang over the bottom elevation pad. (If legs are very long or short, you may take the elevation pad and move it forwards or backwards.)
- For closing pull the hook and loop fastener straps (C) over the leg towards the D-rings. Thread the straps through the D-rings. Use the hook and loop fastener closures to secure the straps. Leave foam flaps slightly open to provide additional ventilation.
- Test for the proper fit by trying to remove the closed extension boot. In case of free movement of the boot the spare pad may be applied in the toe area by removing the adhesive backings only from the inner side of the suspension padding. Your fingers should be able to fit comfortably between the heel opening and the bed. The boot should not be placed neither too loose nor too tight around the leg. The hook and loop fastener straps should not be in contact with the skin on top of the leg.
- In order to prevent lateral rotation of the foot use the extra pad and place it on the left or right side along with the tibia. Remove the adhesive backing and press the pad laterally onto the suspension padding.

Application of traction device

- Remove the long hook and loop fastener attached traction straps (D) from the sides of the extension boot.
- Unravel the traction rope (A) and double knot one end to the rope attachment bar (B).
- Fix the rope attachment bar with the hook and loop fastener traction straps to both sides of the Heelift® Traction Boot. Place the plate for the traction bar to be outside.
- Pass the unknotted end of the traction rope through the prepositioned traction pulley on the cross bar attached to the frame at the foot of the bed.
- Pull firmly on the rope and create a short 2,5 cm large loop in the rope with a double square knot (not a slip knot) just beyond the pulley.
- Attach a weight hanger with the prescribed weight (max. 4,5 kg).
- Elevate the extension boot together with the traction bar so that the Heelift® Traction Boot and the leg are 0,6 cm above the bed. You should be able to fit your finger between the heel opening and the bed.
- If the leg is externally rotated, apply the spare pad beneath the traction straps on the outside of the traction boot.
- To avoid heel cord contracture, secure the extra pad in a vertical position behind the sole of the foot.

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