Hip-Compression MECRON

USA/GB Technical data

Sizes:

The hip compression stabilisation bandage comes in 2 sizes depending upon circumference. Measure the hip circumference. The bandage comes in two versions for the right and left side.

Size	Circumference
M	95 – 115 cm
L	105 – 125 cm

Colour: White

Material:

Outer side of the bandage: polyamide
Inner side of the bandage: 100% cotton
Tape fasteners: 89% viscose, 11% elastodiene
Velcro fasteners: polyamide

Care: The product is maintenance-free.

Caution

Always follow these instructions for use. If you notice any side effects (e.g. swollen skin, pressure points, disrupted blood circulation, etc.) please consult your doctor. If used improperly, the product's function and optimum protection can no longer be guaranteed.

This appliance is intended to assist you in your recovery and should only be used as prescribed by (or in consultation with) your doctor or medical specialist. DARCO (Europe) GmbH does not assume any responsibility.

Avoid using the bandage in water and exposure to acids and solvents. We recommend that you refrain from driving while you are wearing this appliance. This appliance is a single-use product.

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USA/GB Instructions for use

MECRON Hip Compression
Hip compression stabilisation bandage

The MECRON hip compression stabilisation bandage is used for guiding, immobilising and stabilising the hip joint. The velcro fasteners permit free adjustment both at the thigh and at the hip. Three plastic rods in the hip area accommodated in a bag ensure the required compression.

Recommended use:

1. Open the velcro strips of the hip compression stabilisation bandage.
2. Place the bandage around the patient's pelvis and thigh so that it fits snugly.
3. Attach the plastic rods laterally.
4. First close the velcro fastener at the pelvis.
5. Now you can close the velcro fastener at the thigh.

Attach the bandage in such a way to ensure a snug fit. Ensure not to trap any skin around the velcro fasteners. To prevent constriction, do not over-tighten.

Indications

- > Stabilisation after repositioning a dislocated hip joint
- > Stabilisation after total endoprosthetic replacement
- > Hip injuries
- > Chronic hip instabilities

Contraindications:

- > No contraindications are known.

Cleaning instructions

The MECRON hip compression stabilisation bandage is washable at 30°. Remove the plastic stabilisation rods prior to washing and close the velcro fasteners.

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