# MedSurg

# MedSurg Pediatric

EN Healing Shoe  
DE Verbandschuh  
ES Zapato postquirúrgico  
FR Chaussure postopératoire  
IT Calzatura ortopedica  
NL Verbandschoen  
DA Bandagesko  
SV Förbandssko  
PL But pooperacyjny  
SL Začasni terapevtski čevlji  
HR Ortopedska cipela  
PT Sapato pós-operatório

# DARCO

## EN Instructions for use

MedSurg | Healing Shoe

MedSurg Pediatric | Pediatric Healing Shoe

### Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example.

We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual.

This aid is intended for single-patient use only.

## Purpose

The MedSurg Healing Shoe provides stability and protects the foot against bumps and reduces pressure on it following foot surgery.

## Indications / Contraindications

### Indications

Postoperative for stabilisation and offloading, e.g. foot osteotomies

### Contraindications

Open fractures

## Design features

The upper material is made of breathable polyester material. The shock-absorbing sole is made of TPR rubber. The sole profile facilitates the rolling off movement. The shoe fits left or right equally.

## Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

1. Open the shoe using the hook and loop straps (A, B).
2. Please ensure that the foot is securely positioned in the shoe.
3. The inner shoe tabs can optionally be placed over each other.
4. The shoe can be visually adapted to the left or right foot by the positioning of the removable hook and loop strap (B). The logo must be located on the outside of the foot.
5. Fasten the shoe using the hook and loop straps. If needed, the length of the upper hook and loop strap (A) can be customised to fit.

## Technical data

### Sizes

**3 women's sizes:** WS (34–36.5), WM (37–38.5), WL (39–41)

**4 men's sizes:** MS (39–41), MM (41.5–43), ML (43.5–45), MXL (45.5–47)

**1 child size (MedSurg Pediatric):** Universal (up to size 32)

### Colour

Black

### Material

<b>Outsole:</b>	TPR
<b>Insole:</b>	EVA
<b>Upper material:</b>	Polyester
<b>Hook and loop straps:</b>	Nylon

## Combination with other products

- > Puzzle Insole®
- > PegAssist® Insole
- > PegContour® Insole
- > Relief Contour Insole
- > PegContour® Insole Uni
- > Toe Cap

## Maintenance

The product does not require maintenance.

## Cleaning instructions

### Hand wash

- > If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- > Leave the shoe to dry at room temperature.

## Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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DARCO International, Inc.