USA / GB Technical data

Sizes:
3 Women sizes:
S (34 – 36,5), M (37 – 38,5), L (39 – 41)

4 Men sizes:
S (39 – 41), M (41,5 – 43), L (43,5 – 45), XL (45 – 48)

1 Children size (MedSurg™ Original Pediatric):
Universal (up to size 31)

Material:
Outsole: TPR
Insole: EVA
Upper material: Foam Polyester Mesh
Velcros®: Nylon

Maintenance:
The product does not require maintenance.

Construction features:
The upper part is made of a breathable nylon fabric which may be stretched lengthwise and crosswise, the rubber sole is made of TPR with shock absorbing effect. The sole profile enables an easier rolling-off movement.

Cleaning instructions

Hand washing:
> When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.
> Hand washing: the MedSurg™ Original may be hand washed in cold water with mild soap and may be air dried afterwards. Please do not use strong solvents for cleaning (alcohol etc.).
> The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.

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**MedSurg™ Original**
Post-op shoe

The shoe fits right and left equally.

1. Open the shoe by means of the velcro® straps (A).
2. Please ensure that the foot is securely positioned in the shoe.
3. The shoe can be customized to the right or left foot by individually adjusting the velcro® straps (B).
4. Pull the velcro® strap to the left or right with the short end of the strap situated at the outside of the shoe. Place the strap over the flap and fasten it accordingly. The inner shoe flaps may overlap each other arbitrarily (B).
5. The long end of the velcro® strap at the inner side of the shoe is now taken back over the flap and fixed onto the velcro® fastener at the outside of the shoe. (C)
6. The velcro® strap may now be shortened / cut off individually.

Indications

> Postoperative stabilization and off-loading pressure
e.g. foot osteotomies

Contraindication:
> Open fractures

Caution

While wearing this shoe do not attempt to drive. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usually. Recommendation to wear bandages or socks in the shoe.

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