Medburg Pro®

DARCO (Europe) GmbH  
Gewerbegebiet 18  
82399 Raisting, Germany  
Telefon +49 (0) 88 07 92 28-0 | Fax -22  
info@darco.de, www.darco.de

**Manufactured by:**

DARCO Int. Trading (Shanghai) LTD  
No. 128 Lane 3458 Hu Min Road  
Shanghai 201108, China

**USA/GB Technical data**

**Sizes:**  
3 Women sizes:  
WS (34 – 36,5), WM (37 – 38,5), WL (39 – 41)

4 Men sizes:  
MS (39 – 41), MM (41,5 – 43), ML (43,5 – 45), MXL (45 – 48)

**Colour:** Black  
Low and high version (extending the ankle)

**Material:**  
Outsole: TPR  
Insole: EVA  
Upper material: Foam Polyester Mesh  
Lining: Brushed Polyester  
Velcros®: Nylon

**Maintenance:**  
The product does not require maintenance.

**Construction features:**  
The upper part is made of a breathable nylon fabric which may be stretched lengthwise and crosswise, the rubber sole is made of TPR with shock absorbing effect. The sole profile enables an easier rolling-off movement.

**Cleaning instructions**

**Hand washing:**  
> When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.  
> The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.

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**MedSurg Pro®**

Post-op shoe

The MedSurg Pro® healing shoe provides stabilisation, shock absorbtion and offloading pressure after foot surgery. The shoe fits right and left equally.

1. Open the shoe by means of the velcro® straps (A, B).
2. Please ensure that the foot is securely positioned in the shoe.
3. The shoe can be customized to the right or left foot by individually adjusting the velcro® straps (A, B).
4. Pull the upper velcro® strap A to the left or right side of the shoe so that the hook strap is located at the outside of the shoe on both sides. Place it over the flap and fasten accordingly. If necessary, please shorten the upper velcro® strap individually.
5. The inner shoe flaps may overlap each other arbitrarily (C).
6. Close the shoe with the lower velcro® strap B so that the DARCO Logo is located at the outside of the shoe.

**Indications**

> Postoperative for stabilisation and offloading, e. g. foot osteotomies

**Contraindication:**  
> Open fractures

**Caution**

While wearing this shoe do not attempt to drive. Use extreme caution when climbing stairs (e. g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usually. Recommendation to wear bandages or socks in the shoe.

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