MedSurg Pro[®]

EN Healing Shoe
DE Verbandschuh
ES Zapato postquirúrgico
FR Chaussure postopératoire
IT Calzatura ortopedica
NL Verbandschoen
DA Bandagesko
SV Förbandssko
PL But pooperacyjny
SL Začasni terapevtski čevlji
HR Ortopedska cipela
PT Sapato pós-operatório

DARCO

EN Instructions for use

MedSurg Pro® | Healing Shoe

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example.

We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual.

This aid is intended for single-patient use only.

Purpose

The MedSurg Pro® Healing Shoe provides stability and protects the foot against bumps and reduces pressure on it following foot surgery.

Indications / Contraindications

Indications

Postoperative for stabilisation and offloading, e.g. foot osteotomies

Contraindications

No contraindications are known.

Design features

The shoe upper is made of a breathable fabric that can be stretched lengthways and sideways, and the slip-resistant sole is made of TPR. The sole profile facilitates the rolling off movement. The shoe has a universal fit for right and left.

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

EN Use

1. Open the shoe using the hook and loop straps (A, B).
2. Please ensure that the foot is securely positioned in the shoe.
3. The shoe can be customized to the right or left foot by individually adjusting the hook and loop straps (A, B).
4. Position the upper hook and loop strap A to the left or right so that the hook strap on both sides is on the outside of the shoe. Fold it around the tab and secure it firmly. If needed, the length of the upper hook and loop strap can be customized to fit.
5. The inner shoe tabs can optionally be placed over each other (C).
6. Close the shoe with the lower hook and loop fastener B such that the DARCO logo is on the outside of the shoe.

Technical data

Sizes **3 women's sizes:** WS (34–36.5), WM (37–38.5), WL (39–41)
4 men's sizes: MS (39–41), MM (41.5–43), ML (43.5–45), MXL (45.5–47)

Colour Black | Low and high (ankle-spanning) version

Material **Outsole:** TPR
Insole: EVA
Upper material: PU foam, polyester
Liner: Polyester
Hook and loop straps: Nylon

Combination with other products

- > Puzzle Insole®
- > PegAssist® Insole
- > PegContour® Insole
- > Relief Contour Insole
- > PegContour® Insole Uni
- > Toe Cap

Maintenance

The product does not require maintenance.

Cleaning instructions

Hand wash

- > If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- > Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner.
Dispose of this product after use in accordance with applicable local laws and regulations.

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