OPTIMA/SBITM Insole

Distributor
 DARCO (Europe) GmbH
 Gewerbegebiet 18
 82399 Raisting, Germany
 Telefon +49 (0) 88 07. 92 28-0 | Fax -22
 info@darco.de | www.darco.de

 OPTIMA MOLLITER SRL
 Via Breda 19/21
 I-62012 Civitanova Marche (MC)
 Telefon +39 07 33. 80 10-60 | Fax -48
 info@optimamolliter.com
 www.optimamolliter.com

DE Technische Daten

5 Größen:
 S (34 – 36), M (37 – 39), L (40 – 42), XL (43 – 45), XXL (46 – 48)

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5 Sizes:
 S (34 – 36), M (37 – 39), L (40 – 42), XL (43 – 45), XXL (46 – 48)

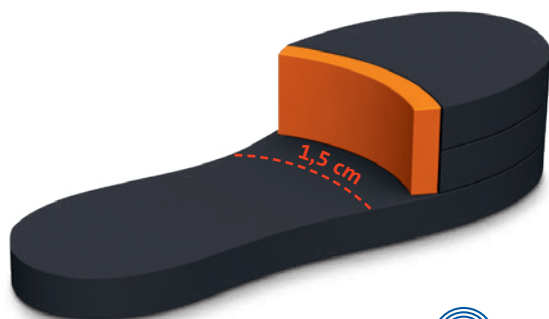
Right and left fit.

Maintenance:
 The product does not require maintenance.

- All required sizes available for the following foot and ankle products:**
- > OPTIMA Diab – Lower leg foot orthosis
 - > OPTIMA PostOP – High Wound Care Shoe
 - > OPTIMA Europa – Wound Care Shoe
 - > SBI Frame – Lower leg foot orthosis
 - > SBI W-Heel – Lower leg foot orthosis

Caution

- > It is recommended to wear bandages or socks.
- > If the product is improperly handled its function and optimal safety can no longer be guaranteed.
- > This aid is intended for single-patient use only.

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USA / GB Instructions for use**OPTIMA / SBi TM Insole**

Transmetatarsal amputation insert

The OPTIMA / SBi TM Insole is designed to protect the amputated foot and enhance its positioning in the shoe. It comprises three different parts connected to one another by adhesive strips and hook and loop.

1. Remove the adhesive strip on the black hook and loop strip and attach it to the inner side of the distal sole block.
2. Attach the hook and loop side of the orange foam pad to the hook and loop side of the black tape.
3. Position the stump of the patient's foot in the shoe in such a way that there is about 1.5 cm clearance between the stump and the distal sole block.

Indications

- > for therapy after transmetatarsal amputation

Contraindication:

- > none known

Cleaning instructions

The sole can be cleaned with warm water and soap or regular disinfectants.