# OrthoWedge OrthoWedge Pediatric

EN Off-loading Shoe  
DE Vorfußentlastungsschuh  
ES Zapato de descarga del antepié  
FR Chaussure de décharge de l'avant-pied  
IT Calzatura per lo sgravio dell'avampiede  
NL Voorvoethulpschoen  
DA Forfodsaflastningssko  
SV Avlastningssko för framfoten  
PL Obuwie odciążające przednią część stopy  
SL Čevlji za razbremenitev sprednjega dela stopala  
HR Čipela za rasterećenje prednjeg dijela stopala  
PT Sapato de alívio do antepé

**DARCO**

## EN Instructions for Use

**OrthoWedge** | Off-loading Shoe

**OrthoWedge Pediatric** | Off-loading Shoe for children

### Important notes

Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or broken skin. You should use bandages or socks, for example.

Do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual.

This aid is intended for single-patient use only.

## Purpose

Off-loading shoe following surgery:

Specifically designed for relief and protection of the forefoot

## Indications / Contraindications

### Indications

- > Postoperative use after surgery to correct hallux valgus or hammer / claw toes
- > Healing of forefoot ulcerations

### Contraindications

- > Open fractures

## Design features

The off-loading shoe encloses the heel and tarsus with a soft padded ankle support. The special sole construction provides relief for the forefoot. The heel cap, hook and loop straps and fasteners prevent slipping in the shoe. The forefoot area is protected against inadvertent impact. The insole is removable and can be replaced with a DARCO insole.

## Combination with other products

- > PegContour® Insole Uni
- > PegAssist® Insole
- > Toe Cap
- > TwinShoe®

## EN Use

1. Open the shoe (A).
2. Step into the open shoe.
3. Please ensure that the foot is securely positioned in the shoe.
4. Close the shoe with the hook and loop straps (A). The foot must not be too tightly or too loosely positioned in the shoe.

### Note

5. When walking, only roll to the inner edges (B). Rolling to the outermost edge (C) can impair the surgical outcome.
6. For prolonged periods of wear, height balancing (TwinShoe®) on the other foot is recommended.

## Technical data

### Sizes

#### 5 adult sizes (OrthoWedge):

XS (33.5–37.0), S (37.5–39.5), M (40.0–41.5),  
L (42.0–44.0), XL (44.5–47.0)

#### 1 child size (OrthoWedge Pediatric):

Universal (up to size 31)

The shoe fits left or right equally.

### Material

<b>Outsole:</b>	TPR
<b>Insole:</b>	EVA
<b>Upper material:</b>	Polyester
<b>Hook and loop straps:</b>	Nylon
<b>Colour:</b>	Black

## Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or medical provider.

## Maintenance

The product does not require maintenance.

## Cleaning instructions

### Hand wash

- > If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- > Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.

## Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. The product can be disposed of with household waste.

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DARCO International, Inc.