USA/GB Instructions for use

PegContour® Insole Uni
Off-loading Cushion Insole

The PegContour® Insole Uni is an effective, three-dimensional insole system for redistribution of plantar pressure. Removal of single pegs enables customization – postoperatively or for acute wound care. A thin layer made of EVA (sole base) stabilises the off-loading area, and keeps the remaining pegs in place. The pressure is reduced by up to 45%.

Recommendations for use

1. Remove the grey or black original sole from the shoe (fig. 1).
2. Determine the area of the foot that needs relief (fig. 2).
3. Carefully remove the pegs at the site at which the pressure should be relieved (fig. 3). Use a suitable tool for this or remove the pegs by hand.
4. Remove the anti-adhesive paper and carefully affix the complete surface of the sole base (supplied) to the pegs. Thus, the pegs are fixed around the pressure-relief zone. This ensures that the area needing relief does not shift (fig. 4).
5. Reinsert the adapted PegContour® Insole Uni in the shoe with the affixed sole base facing downwards (fig. 5).
6. Fitting finished.

Indication

- > for use after hallux valgus and small toe surgeries (Austin, Akin, Scarf, Weil, Hohmann, etc.)
 - > angioneuropathic changes of the foot in combination with mild foot deformities
 - > healing of ulcers in the heel area (rearfoot ulcers)
 - > heel spurs (plantar fasciitis)
 - > heel wounds
 - > postoperative use and treatment of the heel
- For selective off-loading for:
- > plantar lesions
 - > plantar ulcers / ulcer prevention
 - > pressure points

Contraindication

- > marked foot deformities of the diabetic foot

Cleaning instructions

The sole can be cleaned with water at approx. 30° C or lukewarm water or with conventional disinfectants. Do not dry the sole on a heating unit; instead allow it to dry at room temperature.

Technical data

- Size:**
- > available in five or seven sizes
 - > right and left fit

The insole is available in the corresponding sizes for the following aids with the pegs in one of two colours (blue or black – according to the model):

Sole with blue pegs for:

- > MedSurg, MedSurg Pro® - Healing Shoe
 - > Relief Dual®, Relief Dual® - Plus Off-loading Shoe
 - > Relief Insert® Walker - Lower Leg/Foot Orthosis
- Sole with black pegs for:
- > OrthoWedge, OrthoWedge Light - Off-loading Shoe
 - > HeelWedge® - Off-loading Shoe
 - > Commodus Open® - Open Healing Shoe
 - > SlimLine® - Cast Boot

Material:

The insole with a height of 15 mm is made of polyurethane. The sole covering is made of polyester. The separate sole base is made of EVA (40° Shore).

Maintenance:

The product does not require maintenance.

Construction features:


The PegContour® Insole Uni, made 100 % from PU material, provides perfect pressure redistribution with its heel contour. The lengthwise and crosswise sole covering provides a high level of wearing comfort. Easy removal of single pegs enables selective work. A thin sole stabilises the off-loading area, and keeps the remaining pegs in place.

Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly.

Signs of fatigue can occur in the cover layer when there is a large gap. This appliance, together with a special shoe, is intended to assist you in your recovery and should only be used as prescribed by (or in consultation with) your doctor or medical specialist. DARCO (Europe) GmbH does not assume any responsibility.

Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages, for example. We recommend that you do not drive a vehicle while you are wearing this aid in a special shoe. Take extreme caution and walk more slowly than usual. This aid is intended for single-patient use only.

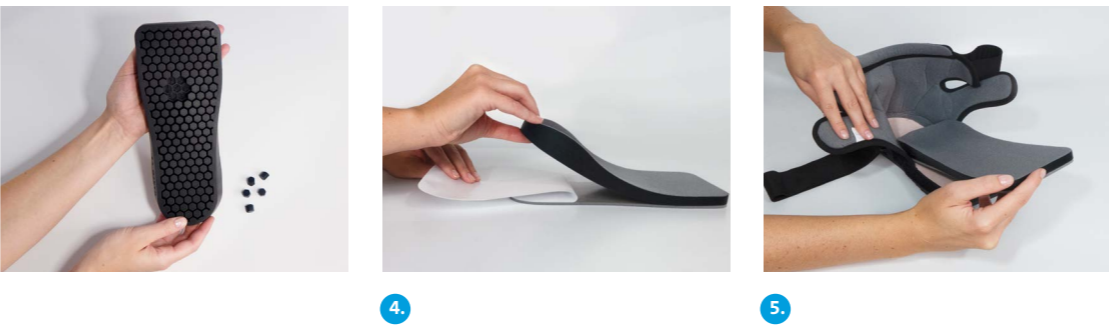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USA/GB
For pressure redistribution and selective off-loading

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Purpose

- > effective insole system for plantar pressure relief in acute care
- > pressure can be reduced by up to 45%
- > the heel contour that adapts to the anatomy of the foot enables optimal pressure redistribution

Features

- > soft lengthwise and crosswise sole covering made of functional polyester
- > customisable PU material
- > customised fit through the removal of single pegs
- > right and left fit

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USA/GB

- Insole with black pegs for:
- > OrthoWedge, OrthoWedge Light - Off-loading Shoe
 - > HeelWedge® - Off-loading Shoe
 - > Commodus Open® - Open Healing Shoe
 - > SlimLine® - Cast Boot

USA/GB

- Insole with blue pegs for:
- > MedSurg, MedSurg Pro® - Healing Shoe
 - > Relief Dual®, Relief Dual® Plus - Off-loading Shoe
 - > Relief Insert® Walker - Lower Leg/Foot Orthosis

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