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DARCO (Europe) GmbH
 Gewerbegebiet 18
 82399 Raisting, Germany
 Telefon +49 (0) 88 07.92 28-0 | Fax -22
 info@darco.de | www.darco.de

Made in China

USA/GB Technical data

The 110° models have been developed for Achilles tendon pathologies (20° plantar flexion).

Sizes:	Sizes:	Sizes:	Foot length of the splint:	Calf section of the splint:
S 90°	S 90° Plus	S 110°	265 mm	340 mm
M 90°	M 90° Plus	M 110°	295 mm	370 mm
L 90°	L 90° Plus	L 110°	325 mm	400 mm

Can be used universally on the right or left foot.

Colour: white-blue

Material:
 Outer shell: Polycarbonate
 Inner liner: PE
 Hook and loop fastener: Nylon

Maintenance:
 The product is maintenance-free.

Caution

Always follow these instructions for use. Contact your doctor immediately if you experience any side effects (such as skin swelling, pressure sores, blood circulation problems, sensory impairment, etc.). The function and optimal protection are no longer guaranteed if the product is incorrectly used. We recommend that bandaging materials be worn to protect the skin. This aid is intended for single-patient use only.

Cleaning instructions

- > Wipe disinfection of the outer shell and inner padding with a surface disinfectant
- > Do not use any substances that contain oil

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Splinting System

Putting on the night splint:

- > First treat the surgical wound or foot injury in the usual manner with appropriate bandaging material.
- > Select the appropriate splint length. The patient's foot and lower leg length is the determining factor for this. On the lower leg, it is important that the splint does not extend too far in the direction of the back of the knee in order to avoid pressure in this area when flexing the knee joint. It should also be ensured that the head of the fibula is exposed in order to avoid pressure damage to the peroneal nerve.
- > Position the foot in the splint. To do this, the heel must be correctly positioned in the heel recess. (Photo 1)
- > Secure the splint using a conventional gauze bandage or cohesive bandage. Hook and loop straps incorporated on the outer shell allow the gauze bandage to be ideally secured (Photo 2). Do not wind too tightly.
- > The bandage material should ideally be approx. 10 cm wide. Begin winding the bandage at the back of the foot and guide the bandage in a distal (away from the torso) to a proximal (towards the torso) direction. In doing so, check that the splint is correctly positioned. (Photos 3 and 4)

Indications

For general use in conservative and postoperative therapy of lower leg, ankle and foot injuries:

- > Postoperative support
- > Post-traumatic stabilization / immobilization
- > Immobilisation following an accident
- > Short-term stabilization (ligament rupture, luxation, fractures, wounds, strains, sprains, etc.)
- > Achilles tendon rupture
- > Preoperative Lagerung

Contraindications:

- > Shortening of the calf muscle
- > Marked Haglund's deformity / posterolateral exostosis
- > Impaired sensation (e.g. polyneuropathy), impaired perfusion
- > Material incompatibility / skin changes (e.g. swelling, redness) due to allergic reactions, inflammation or injury
- > Do not wear directly on open wounds or irritated skin.

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