Relief Dual[®]

Relief Dual[®] Plus

EN Off-loading Shoe
DE Fußteilentlastungsschuh
ES Zapato de descarga
FR Chaussure de décharge partielle du pied
IT Calzatura per lo sgravio parziale del piede
NL Schoen ter ontlasting van bepaalde voetgedeelten
DA Aflastningssko
SV Avlastningssko
PL But odciążający
SL Čevlji za razbremenitev stopala
HR Čipela za rasterećenje dijela stopala
PT Sapato de alívio

DARCO

Relief Dual®

Toe Cap

Relief Dual® Plus

EN Instructions for use

Relief Dual® | Off-loading Shoe

Relief Dual® Plus | Off-loading shoe with high ankle support

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example.

We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual.

This aid is intended for single-patient use only.

Purpose

The Relief Dual® / Relief Dual® Plus off-loading shoe provides stability and protects the foot against bumps and reduces pressure on it following foot surgery.

Indications / Contraindications

Indications

Relief Dual®

- > Postoperative for stabilisation and offloading, e.g. foot osteotomies

Relief Dual® Plus

- > Postoperative for stabilisation and offloading, e.g. foot osteotomies
- > Postoperative due to removal of Haglund exostosis
- > Postoperative due to plantar fasciitis
- > For wound treatment

Contraindications

- > No contraindications are known.

Design features

The upper is made of a breathable fabric. The dual sole technology comprising EVA and rubber is lightweight, abrasion-resistant, shock-absorbent and anti-slip. The low sole height with special profile eases the rolling movement. The insole is replaceable. The shoe can be worn on the left or right.

Maintenance

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Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. The product can be disposed of with household waste.

1. Open the shoe using the hook and loop straps (A, B).
 - 1a. Relief Dual® Plus: If required, you can also open the hook and loop straps C (left and right).
2. Please ensure that the foot is securely positioned in the shoe.
3. Relief Dual® Plus: First close the two hook and loop straps C of the heel cap.
- 3a. The inner shoe tabs can be placed over each other either way (D).
- 3b. Close the two hook and loop straps A and B as described below.
4. The shoe can be customized to the right or left foot by individually adjusting the hook and loop straps (A, B).
5. Guide the upper hook and loop strap A through both straps on the shoe.
6. Fold the strap on the outer side (left or right) around the tab and secure it firmly. Close the shoe with the strap on the other side. If needed, the length of the upper hook and loop strap can be customized to fit.
7. Close the shoe with the lower hook and loop fastener B such that the DARCO logo is on the outside of the shoe.

Toe Cap (optionally available):

The DARCO Toe Cap (E) allows the DARCO shoe to be worn in closed form. The cap protects the foot from cold and dirt and improves comfortable fitting.

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling/cleaning.

Technical data

Sizes

3 women's sizes:

WS (34 – 36.5), WM (37 – 38.5), WL (39 – 41)

5 men's sizes:

MS (39 – 41), MM (41.5 – 43), ML (43.5 – 45), MXL (45.5 – 47), MXXL (47.5 – 50)

Colour

Relief Dual®: Grey

Relief Dual® Plus: Grey

Material

Outsole: Rubber, EVA

Insole: EVA

Upper material: Polyester foam, microfibre

Inner lining: Polyester

Hook and loop straps: Nylon

Adhesive: Polyurethane glue

Combination with other products

- > Puzzle Insole®
- > PegAssist® Insole
- > Relief Contour Insole
- > PegContour® Insole (blue)
- > PegContour® Insole Uni (blue)
- > Toe Cap

Cleaning instructions

Hand wash

- > If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- > Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.

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