+ SlimLine® Pediatric

DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Toll-free 800.999.8866
Phone 304.522.4883
Fax 304.522.0037
www.darcointernational.com

CE REP
DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 (0) 88 07.92 28-0
Fax +49 (0) 88 07.92 28-22
info@darco.de
www.darco.de

Made in China

USA / GB Technical data

Sizes:
5 Adult sizes (SlimLine®):
XS (33,5 – 37), S (37,5 – 39,5), M (40 – 41,5), L (42 – 44), XL (44,5 – 47)

3 Children sizes (SlimLine® Pediatric):
S (up to size 21), M (22 – 26), L (27 – 32)

The shoe fits right and left equally.

Colours:
Adult model (SlimLine®): Black
Children's model (SlimLine® Pediatric): Navy or Pink-Navy

Material:
Outsole: TPR
Insole: EVA
Upper material: EVA / Polyester
Hook and loop fastener straps: Nylon

Maintenance:
The product does not require maintenance.

Construction features for adult and children's model:
The upper part consists of a breathable EVA / polyester material. The rubber sole is made of TRP with shock absorbing effect. The sole profile enables an easier rolling-off movement.

Cleaning instructions

Hand washing:
> When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.
> The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.

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Tamanhos:
5 Tamanhos de adulto (SlimLine®):
XS (33,5 – 37), S (37,5 – 39,5), M (40 – 41,5), L (42 – 44), XL (44,5 – 47)

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SlimLine® / -Pediatric

Cast Boot

1. Open the boot (A).
2. Enter into the opened boot.
3. Please ensure that the foot is securely positioned in the boot and that the heel area of the foot is as far back as possible in the boot.
4. Close shoe with the hook and loop fasteners (A). The foot should be placed comfortably in the boot, not too loose and not too tight.

Optional:

The inner sole is to be removed / replaced by a healthcare specialist.

Indications

> Fracture and cast care

Caution

Your physician has prescribed you a very unusual looking shoe. Its design allows for the most rapid healing of fractures and recovery after surgery with unlimited mobility at the same time. Walking with this boot is more difficult than walking with conventional footwear; please read the following precautions.

Do not attempt to drive while wearing this boot. Use extreme caution when climbing stairs, stepping up onto curbs and walking on uneven surfaces. Walk more slowly than usual. You are not wearing a normal street shoe and more clearance is required for the wedge sole. Excessive weight bearing (edematous patients) may cause a breakdown of the shoe's functional sole. In case the sole must be fixed due to extensive wearing, please contact your physician or orthopedic technician. Recommendation to wear with bandages or socks.

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