TAS[®]

EN Hallux Valgus Toe Alignment Splint
DE Hallux Valgus Zehenschiene
ES Férula para dedo hallux valgus
FR Attelle de correction – Hallux Valgus
IT Tutore per alluce valgo
NL Hallux valgus teenspalk
DA Tåskinne til knyster
SV Hallux valgus tåskena
PL Hallux Valgus Szyna na palce
SL Opornica za izkrivljen nožni palec
HR Odstojnik za nožne prste za hallux valgus
PT Tala de dedo para Hallux Valgus

DARCO

MD EN Medical Device
DE Medizinprodukt
ES Dispositivo médico
FR Dispositif médical
IT Dispositivo medico
NL Medisch hulpmiddel
DA Medicinsk udstyr
SV Medicinteknisk produkt
PL Urządzenie medyczne
SL Medicinski pripomoček
HR Medicinski proizvod
PT Dispositivo médico

UDI EN Unique Device Identifier
DE Produktidentifizierungsnummer
ES Identificador único del producto
FR Identifiant unique des dispositifs
IT Identificatore univoco del dispositivo
NL Unieke identificatie van het hulpmiddel
DA Unik identifikator på udstyret
SV Unik produktidentifiering
PL Unikalny identyfikator urządzenia
SL Edinstveni identifikator pripomočka
HR Jedinstveni identifikator proizvoda
PT Identificador único do dispositivo

 EN Latex-free
DE Latexfrei
ES Sin látex
FR Sans latex
IT Senza lattice
NL Latexvrij
DA Latexfrit
SV Latexfri
PL Niezawierające lateksu
SL Brez lateksa
HR Ne sadrži lateks
PT Sem látex

 EN Single patient multiple use
DE Einzelner Patient, mehrfach anwendbar
ES Reutilizable en un solo paciente
FR Usage multiple pour patient unique
IT Uso multiplo su un solo paziente
NL Meermalig gebruik bij één patiënt
DA Flergangsbrug til en enkelt patient
SV För användning flera gång av en patient
PL Wielokrotnego użytku przeznaczone dla jednego pacjenta
SL Večkratna uporaba pri enem bolniku
HR Za višekratnu upotrebu na jednom pacijentu
PT Várias utilizações num único doente

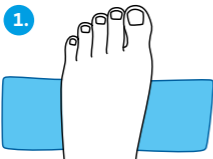
DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting | Germany
Tel. +49 8807 9228-0
Fax +49 8807 9228-22
info@darco.de | www.darco.de

Made in EU

2021-10-28 Rev. 01

EN Metatarsal band
DE Metatarsalband

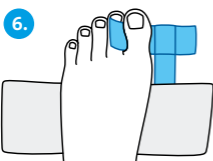
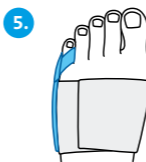
EN Toe loop
DE Zehenschlaufe

EN Heel band
DE Fersenband


EN Toe loop / DE Zehenschlaufe

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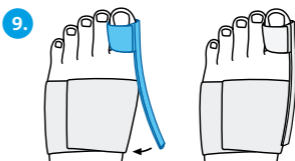Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities. Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product on injured, irritated, or damaged skin. You should use bandages, for example. While wearing this aid, ensure sufficient forefoot clearance and avoid all kinds of outside mechanical influences. This aid is intended for single-patient use only.

Purpose

The bandage is for post-operative use to maintain the surgical result.

Indications / Contraindications

Indications

Post-operative use after:

- > Hallux valgus surgery
- > Tailor's bunion surgery
- > Hammer toe surgery

Contraindications

No contraindications are known.

Design features

The TAS® Toe Alignment Splint is a postoperative bandage for alignment that is used instead of taping. It enables positioning and alignment of toes in the correct position after surgery/correction, which has a decisive impact on the surgical result. A sandal strap at the back of the foot reduces slippage of the midfoot bandage. The soft material makes it comfortable to wear.

Use

Hammer toes (fig. 1 – 3):

The elastic metatarsal band is designed so that it can be adjusted to the foot individually. Place the band around the foot as shown (fig. 1 – 2).

The heel strap prevents the bandage from slipping and is put on as shown in fig. 3.

Tailor's bunion (fig. 4 – 5):

The small toe loops are fixed to the metatarsal band according to need using the hook and loop strap (fig. 4 – 5).

Hallux valgus (fig. 6 – 9):

The big toe loop is fixed as shown in fig. 6 – 7.

The stabilising element can be inserted into the big toe loop (fig. 8 – 9).

Combination with other products

No specifications

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling/cleaning.

Technical data

Size:

Universal
Can be worn on the right and left

Colour:

White

Material:

Elastic strap: Polyester, polyamide, viscose, elastane
Hook strap: Polyamide
Fleece strap: Polyamide
Velour: Polyamide
Tag: Polyester satin
Label: Polyester satin
Stabilising element: Polyethylene, polyester
Sewing thread: Polyester

Maintenance

The product does not require maintenance.

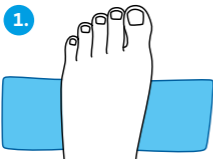
Cleaning instructions

- > Hand wash (cold or lukewarm, max. 30°C)
- > Do not bleach
- > Do not dry in dryer
- > Do not iron
- > Do not dry-clean

Disposal

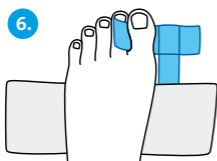
Dispose of after use. Please dispose of the packaging in an environmentally manner. The product can be disposed of with household waste.

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