# DARCO Toe Cap

WCS®, OrthoWedge®, TwinShoe®, MedSurg/-Pro®, Relief Dual®

DARCO International, Inc.  
810 Memorial Blvd.  
Huntington, WV 25701, USA  
Toll-free 800.999.8866  
Phone 304.522.4883  
Fax 304.522.0037  
www.darcointernational.com

DARCO (Europe) GmbH  
Gewerbegebiet 18  
82399 Raisting, Germany  
Telefon +49 (0) 88 07.92 28-0 | Fax -22  
info@darco.de, www.darco.de

**USA/GB Application**

> Protection from cold and dirt

**Technical data**

**Sizes:**  
Available in all sizes for the products:  
WCS®, OrthoWedge, TwinShoe®, MedSurg/-Pro® and Relief Dual®

Right and left fit. (OrthoWedge, TwinShoe®, MedSurg/-Pro®, Relief Dual®)  
Available for right and left shoe. (WCS®)

**Colour:** Black, Grey (Relief Dual®)

**Material:**  
WCS®: Spandex / Polyester  
OrthoWedge, TwinShoe®, MedSurg: Dacron Polyester  
MedSurg Pro®: Foam Polyester Mesh  
Relief Dual®: Polyester Foam, Microfibre

**Maintenance:**  
The product does not require maintenance. The DARCO Toe Cap is designed for the use of one shoe only.

**Cleaning instructions**

The DARCO Toe Cap is only to be hand washed.

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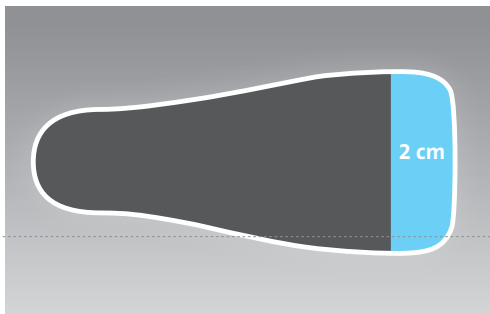
## USA / GB Instructions for use

### DARCO Toe Cap Toe cap

The DARCO Toe Cap (fig. 1) allows the DARCO shoe to be worn in closed form. The cap protects the foot from cold and dirt and improves comfortable fitting.

#### Recommendations for use:

1. Take out the removable insole from the orthopedic shoe.
2. Remove the hook and loop fastener from the toe cap and take off the white adhesive strip. (fig. 2)
3. Then stick the hook and loop fastener parallel to the front edge of the sole onto the inner side of the running sole. Please see to it that the hook and loop fastener is not further away from the front sole edge than 2 cm maximum. (fig. 3)
4. Now insert the insole plus toe cap into the shoe.
5. Press onto the insole with your hand in order to ensure secure fixing and close the shoe as usual. (fig. 4)

### Caution

This aid is complementing the orthopedic shoe and protects the foot from cold and dirt. If you experience pain, abnormal swelling, noticeable changes or other unusual reactions, discontinue use and contact your physician or healthcare specialist immediately. In case of improper handling function and maximum safety of the product can no longer be guaranteed. This aid is intended for single-patient use only.

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