TwinShoe®

EN Height compensating shoe
DE Höhenausgleichsschuh
ES Zapato compensador de altura
FR Chaussure de compensation de hauteur
IT Calzatura di compensazione dell'altezza
NL Hoogtecompensatieschoen
DA Udligningssko
SV Höjdhöjningsssko
PL Obuwie ortopedyczne do wyrównania wysokości
SL Čevlji za izravnavo višine
HR Čipela za izjednačenje visine
PT Sapato compensador da altura

DARCO

EN Instructions for use

TwinShoe® | Height compensating shoe

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.), please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a health-care professional. DARCO (Europe) GmbH does not assume any responsibility.

Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use bandages, dressings or socks.

Do not drive a vehicle while you are wearing this product. Use extreme caution when walking with this device on uneven surfaces. Walk more slowly than usual.

This device is intended for single-patient use only.

Purpose

The TwinShoe® is a height compensating shoe for our proven products. The DARCO TwinShoe® guarantees a safe and physiologically correct gait.

Indications / Contraindications

Indications

- > Compensation of leg length difference (approx. 45 mm)
- > Avoidance of pelvic tilt

Contraindications

Open fractures

Design features

The upper is made of a breathable nylon fabric that can be stretched lengthways and sideways, and the sole is made of shock-absorbing TPR rubber. The sole profile facilitates the rolling off movement.

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling/cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. The length of time this device is worn depends on the indication. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

EN Use

1. Open the shoe (A).
2. Step into the open shoe.
3. Make sure that the foot is positioned securely in the shoe and the heel is as far back as possible in the shoe.
4. Close the shoe using the hook-and-loop straps (A). The foot must not be too tightly or too loosely positioned in the shoe.

Technical data

Sizes	5 sizes XS (33.5 – 37), S (37.5 – 39.5), M (40 – 41.5), L (42 – 44), XL (44.5 – 47) Universal fit for right and left
Colour	Black
Material	Outsole: TPR Insole: EVA Upper material: Polyester

Combination with other products

Can be used with the following DARCO products

- > WCS®
- > SBI Frame
- > OPTIMA PostOp
- > OPTIMA Europa
- > OPTIMA Free
- > OrthoWedge
- > OrthoWedge Light
- > HeelWedge®

Maintenance

The product does not require maintenance.

Cleaning instructions

Hand wash

- > If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- > The TwinShoe® can be hand washed in cold water with a mild soapy solution. Please do not use any aggressive cleaning agents (alcohol etc.).
- > Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.

Disposal

 Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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V-M Orthotics Ltd.
ALBO-Healthcare GmbH