TwinShoe®

USA / GB Technical data

5 Sizes:
XS (33.5 – 37), S (37.5 – 39.5), M (40 – 41.5), L (42 – 44), XL (44.5 – 47)

The shoe fits right and left equally.

Material:
Outsole: TPR
Insole: EVA
Upper material: Polyester

Maintenance:
The product does not require maintenance.

Construction features:
The upper material consists of breathable polyester material. The rubber sole is made of TRP with shock absorbing effect. The sole profile enables an easier rolling-off movement.

Can be used with the following DARCO products:
WCS®
SBI
OPTIMA PostOp
OPTIMA Europa
OPTIMA Free
OrthoWedge
OrthoWedge
HeelWedge®

Cleaning instructions

Hand washing:
> When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.
> Hand washing: The TwinShoe® may be hand washed in cold water with mild soap and may be air dried afterwards. Please do not use strong solvents for cleaning (alcohol etc.).
> The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.

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- Open the boot (A).
- Enter into the opened boot.
- Please ensure that the foot is securely positioned in the boot and that the heel area of the foot is as far back as possible in the boot.
- Close shoe with the hook and loop fastener (A). The foot should be placed comfortably in the boot, not too loose and not too tight.

Indications

- › Balance shoe for postoperative care following forefoot and midfoot surgery.
- › When prescribing aids which cause a potential difference in leg lengths (such as a forefoot off-loading shoe).
- › The TwinShoe® helps avoid pelvic tilt and can also be worn outside of enclosed spaces. The length of time this aid is worn depends on the indication.

- Contraindication:**
- › Open fractures

Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings or socks with the shoe. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for single-patient use only.

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