USA/GB Indication

- > For treatment of diabetic foot syndrome
- > For healing of open wounds and minor ulceration
- > For pressure offloading
- > Postoperative after toe and forefoot surgery

Contraindication

- > Open fractures

Technical data

Size:
The WCS® Light is available pairwise in seven sizes.

Material:
Outsole: PU
Upper material: Spandex / Polyester

Maintenance:
The product does not require maintenance. The WCS® Light is designed for the use of one person / patient only.

Plantar layer system:

Material	Density	Thickness	Durometer*
EVA	soft	6.3 mm	26
EVA	medium	6.3 mm	42
EVA	firm	6.3 mm	50
Insole foam	high	3.1 mm	29

*The insole durometers may vary within 2" to 3" of those listed.

Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings or socks with the shoe. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for single-patient use only.

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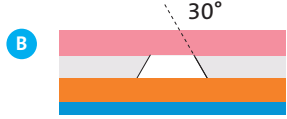
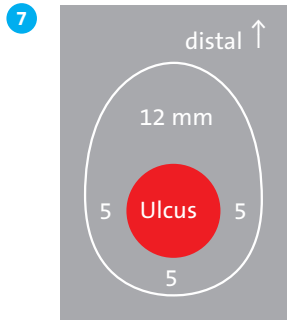
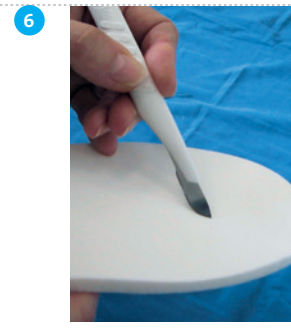
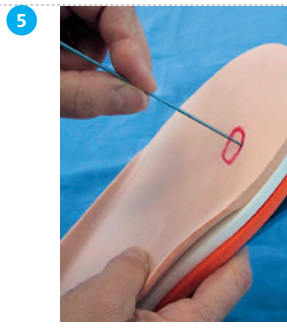
WCS® Light

Closed wound care shoe

The wound care shoes are used in pairs. Determining the correct shoe size is important for the perfect functionality of the shoe. The hook and loop closure system enables to easily and comfortably open and close the wound care shoes. The WCS® Light system in particular supports patients limited in their mobility. For a proper stabilization of the foot the hook and loop straps should be closed accordingly. The removable insoles are already included within the shoe (fig. 1). The proven four-layer-insoles are part of the shoe system (fig. 2) and can be customized individually (fig. 6).

Marking the ulceration and customizing the insoles:

- Place a sterile film over the wound or pressure ulcer (fig. 3) and draw a circle on the film around the affected area with the marker pen supplied (fig. 4).
- The patient should now stand up in the shoe /s onto the insoles which then pictures the ulcer onto the upper layer of the insole.
- Puncture the two layers following the upper layer by penetrating this specific layer with a sharp tool. Then mark the area to be cut out (fig. 5)
- Draw the areas to be removed onto the lower side of the layers as described in fig. 7.

Recommendations for use:

- > The light wound care shoe is specifically used for minor ulcers. The period of application depends on the grade of ulceration.
- > Daily wear with indoor and outdoor use.
- > Recommendation to wear bandages or socks.

Insole cutting:

- A No plantar open ulcers
 B Level of ulceration 0 and 1, Wagner Scale, open wounds, bevel cut with approx. 30°
 C After wound healing (of B)
 D Level of ulceration 2 and up, Wagner Scale, open wound
 E After wound healing (of D)
 F Cross section depicting offloading lesion under third metatarsal head

Cleaning instructions

When slightly dirty clean the WCS® Light shoes with a damp cloth or a soft brush. Please do not use strong solvents for cleaning (alcohol etc.). Do not tumble dry or on the stove. Allow to air dry. The enclosed insoles can be washed with warm water and soap or cleaned with regular disinfectants.

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