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# YDA Your Daily Activity

## Cleaning instructions

The product is cleaned as follows:  
 > Under no circumstances may the shoe be immersed in water, placed under a water jet or washed in a washing machine.  
 > Use a soft sponge moistened with lukewarm water (maximum 40° C) and neutral soap to clean the bottom side surface of the sole.  
 > Do not use liquid alcohols, liquid abrasive cleaners, detergents, alcohol, laundry detergent, bleach, ammonia, solvents. Preferably, cleaning agents with good biodegradability should be used.  
 > Do not use abrasive agents or tools, brushes, steel wool, glass paper, blades or the like to remove stains or residues.  
 > Do not dry the shoe in a dryer or on a radiator. Let it dry in the air at room temperature.

It is important not to alter the designated support points so that the functionality of the product is not affected.  
 The comfort shoe can be used directly with the supplied insole, or can accommodate a suitable orthotic insole (with a maximal height of 8 – 10 mm) made by a licensed healthcare practitioner, replacing the one supplied as standard in the footwear.

**Maintenance:** The product requires no maintenance.  
 Only materials that have not been chemically treated with prohibited products were used for the lining. Nevertheless, due to the normal perspiration of the foot, discoloration and / or staining may occur, which is perfectly normal.  
**Material:**  
 Upper: Mesh  
 Outsole: Rubber / Pylon  
 Plantar insole: Polyurethan, Synthetic fabric, Nylon  
 Insole: Texon, Synthetic fabric  
 Inner lining: Synthetic fabric  
 Laces: Elastic Laces + Fabric Laces  
**Colour:** The color range can vary according to the stock availability.

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### YDA

Comfort Shoes

- > Probe the interior of the shoes each time before you put them on and remove any extraneous matter.
- > Never wear the comfort shoes without the supplied insoles or a special inlay from your orthopaedic retail specialist.
- > To avoid pressure sores and vascular disorders do not lace or fasten the shoe too tightly.
- > The comfort shoes have been developed and designed according to state-of-the-art technology. They comply with Directive 93 / 42 EEC.

## Indications

**After an evaluation by a specialized physician, the YDA shoes can be suggested for the following management:**

- > Metatarsalgia
- > Sesamoiditis
- > Flexor and extensor tendon contractures
- > Hallux limitus and rigidus
- > Hallux valgus
- > Claw and hammer toes
- > Tailor's bunions
- > Diabetic foot in primary prevention of risk level 0 (no neuropathy or vasculopathy)
- > Diabetic foot in primary prevention of risk level 1 (neuropathy, vasculopathy)
- > Diabetes-patients with risk level 2 (neuropathy and vasculopathy with deformation)

### Postoperative

- After post-operative care of approx. 6 weeks (stable healing) with a post-op shoe for:
- > Hallux valgus corrections and corrective osteotomies (Austin / Chevron, Akin, Scarf etc.)
  - > Arthrodesis of the metatarsophalangeal joint and first TMT
  - > Osteotomies of the small toes (Weil etc.)

### Contraindications:

- > None known

## Caution / Risks

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings or socks with the shoe. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for singlepatient use only.