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DARCO

Cleaning instructions

The product is cleaned as follows:
 > Under no circumstances may the shoe be immersed in water, placed under a water jet or washed in a washing machine.
 > Use a soft sponge moistened with lukewarm water (maximum 40° C) and neutral soap to clean the bottom side surface of the sole.
 > Do not use liquid alcohols, liquid abrasive cleaners, detergents, alcohol, laundry detergent, bleach, ammonia, solvents. Preferably, cleaning agents with good biodegradability should be used.
 > Do not use abrasive agents or tools, brushes, steel wool, glass paper, blades or the like to remove stains or residues.
 > Do not dry the shoe in a dryer or on a radiator. Let it dry in the air at room temperature.

It is important not to alter the designated support points so that the functionality of the product is not affected.
Construction features:
 The comfort shoe can be used directly with the supplied insole, or can accommodate a suitable orthotic insole (with a maximal height of 8 – 10 mm) made by a licensed healthcare practitioner, replacing the one supplied as standard in the footwear.
Maintenance: The product requires no maintenance.
 Only materials that have not been chemically treated with prohibited products were used for the lining. Nevertheless, due to the normal perspiration of the foot, discoloration and / or staining may occur, which is perfectly normal.
Material:
 Upper: Mesh
 Outsole: Rubber / Phylon
 Plantar insole: Polyurethan, Synthetic fabric, Nylon
 Insole: Texon, Synthetic fabric
 Inner lining: Synthetic fabric
 Laces: Elastic Laces + Fabric Laces
Colour: The color range can vary according to the stock availability.
 7 Woman sizes: 35 – 41
 8 Man sizes: 40 – 47

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USA / GB Instructions for use

YDA
Comfort Shoes

- > Probe the interior of the shoes each time before you put them on and remove any extraneous matter.
- > Never wear the comfort shoes without the supplied insoles or a special inlay from your orthopaedic retail specialist.
- > To avoid pressure sores and vascular disorders do not lace or fasten the shoe too tightly.
- > The comfort shoes have been developed and designed according to state-of-the-art technology. They comply with Directive 93/42 EEC.

Indications

After an evaluation by a specialized physician, the YDA shoes can be suggested for the following management:

- > Metatarsalgia
- > Sesamoiditis
- > Flexor and extensor tendon contractures
- > Hallux limitus and rigidus
- > Hallux valgus
- > Claw and hammer toes
- > Tailor's bunion
- > Diabetic foot in primary prevention of risk level 0 (no neuropathy or PAD)
- > Diabetic foot in primary prevention of risk level 1 (no neuropathy or PAD, with deformity)
- > Diabetes-patients with risk level 2 (neuropathy and PAD)

Postoperative

- After post-operative care of approx. 6 weeks (stable healing) with a post-op shoe for:
- > Hallux valgus corrections and corrective osteotomies (Austin / Chevron, Akin, Scarf etc.)
 - > Arthrodesis of the metatarsophalangeal joint and first TMT
 - > Osteotomies of the small toes (Weil etc.)

Contraindications:

- > None known

Caution / Risks

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings or socks with the shoe. This aid is intended for singlepatient use only.