MECRON ReadyDressment Strips

EN Elastic Stabilising Strips
DE Elastische Stabilisierungszügel
ES Cintas elásticas para la estabilización
FR Brides élastiques de stabilisation
IT Cinghie elastiche stabilizzanti
NL Elastische stabiliserende strips
DA Elastisk stabiliseringsstøje
SV Elastisk stabiliseringsrem
PL Elastyczne plastry stabilizujące
SL Elastične stabilizacijske zanke
HR Elastična stabilizacijska traka
PT Fitas elásticas de estabilização

MECRON

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product on injured, irritated, or damaged skin.

This product is a single-use product.

Purpose

Elastic pre-cut tapes that are ready for immediate use, e.g. after minimally invasive surgical procedures.

Indications / Contraindications

Indications

- > Postoperative and conservative use for the redressement and stabilisation of fingers and wrists, toes and ankles
- > Postoperatively following open foot procedures

Postoperatively following minimally invasive foot procedures:

- > Akin osteotomies
- > Chevron osteotomies
- > DMMO (distal metatarsal metaphyseal osteotomies)
- > Claw toe / hammer toe osteotomies
- > Curly toe osteotomies
- > Digitus quintus varus osteotomies
- > Tailor's bunions osteotomies
- > Tenotomies

Contraindications

- > Large skin injuries
- > Open wounds with risk of infection
- > Uncertain diagnoses
- > Allergic skin reactions to the tape material
- > Thrombosis / vascular disease
- > Arterial or venous occlusive disease
- > Tumour

Design features

Elastic and pre-cut tape strips with a wavy adhesive surface.

EN Use

- > First of all, obtain a clear overview of the indication and fitting.
- > Test the patient for tolerability to the tapes.
- > Remove any body hair at the site of use (optional). Please use a long-hair trimmer for this (do not use a wet razor!).
- > Apply the tape to clean (washed with warm water and conventional soap), dry skin.
- > The ends of the tape should be applied with no stretch, if possible.
- > Apply the tape with the outside of the thumb, making sure that the tape does not arch or form wrinkles.
- > Try not to touch the adhesive layer to prevent removal of the adhesive before the tape is applied.
- > Rub the tape on after applying it. The adhesive is activated through this rubbing and you achieve a longer wearing time.
- > When removing the tape, make sure to fix the surrounding tissue with your hand. This reduces your patient's pain and protects the skin from excessive irritation.

Combination with other products

No specifications

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling.

Technical data

- Sizes:** Short: 10 × 1.4 cm (length stretchable up to 16.5 cm)
Long: 13.5 × 1.8 cm (length stretchable up to 22 cm)
- Colour:** White, blue
- Material:** Cotton, elastane and acrylic adhesive

Maintenance

The product does not require maintenance.

Cleaning instructions

No cleaning instructions.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. The product can be disposed of with household waste.

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TS Co., Ltd.
Emergo Europe B.V.