MECRON ReadyDressment Strips

MECRON

USA/GB Instructions for use

MECRON ReadyDressment Strips

Elastic stabilizing strips

- > A clear overview of indication and disposition is required.
- > Find out whether the patient shows allergic reaction to taping material.
- > Remove body hair around application area (optional). Please use a long-hair trimmer (no wet shaving!).
- > Apply the tape on clean (with warm water and soap) and dry skin.
- > Ensure that ends are laid down with no tension.
- > Attach the tape with the outside of the thumb, making sure that the tape does not arch and that the tape is not creased.
- > Avoid touching the glue layer during use to prevent removal of the adhesive.
- > After attachment of the tape rub it on, to activate the glue and prolong the adhesion.
- > When removing the tape, fix the surrounding tissue with your hand. This reduces your patient's pain and protects the skin from excessive irritation.
- > The MECRON ReadyDressment Strip is a single-use product.

Indications

- > Postoperatively as well as conservative for the redressement and stabilization of wrists, fingers, toes or ankles
- > Postoperatively for surgical foot interventions
- > Postoperatively following minimally invasive surgical techniques:
 - > Akin osteotomies
 - > Chevron osteotomies
 - > DMMO (distal metatarsal metaphyseal osteotomy)
 - > Claw toe / hammer toe corrections
 - > Curly toe reconstructions
 - > Cock-up toe D-V
 - > Tailor's bunion corrections
 - > Tenotomies

Contraindications:

- > large and serious skin injuries
- > open wounds with risk of infection
- > nondistinctive diagnoses
- > allergic reactions to tape material
- > pulmonary veno-occlusive disease
- > thrombovascular disease
- > in the event of cancer or tumors

Technical data

Sizes: Short: 10 × 1.4 cm (Length stretchable up to 16.5 cm)

Long: 13.5 × 1.8 cm (Length stretchable up to 22 cm)

Colours: White, Blue

Material: Cotton and acrylic adhesive, Latex free

Maintenance: The product does not require maintenance.

Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. If the product is handled incorrectly, its function and optimum protection can no longer be guaranteed. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. This device is a single-use product. The clinical efficacy of kinesiology tapes is professionally controversial and not scientifically proven by studies.

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TS Co., Ltd.
Emergo Europe B.V.