
# MECRON X-patch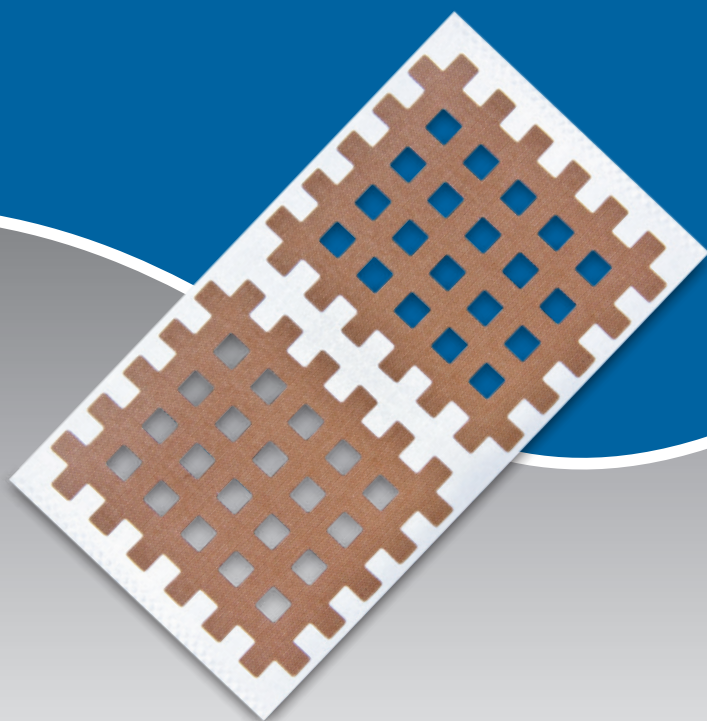

**MECRON**

## USA / GB Instructions for use

### MECRON X-patch

#### Cross Acupuncture Tape

- > To begin with, obtain a clear picture of the condition of the patient being treated.
- > Find out whether the patient shows allergic reaction to taping material.
- > Remove body hair around application area (optional). Please use a long-hair trimmer (no wet shaving!).
- > Apply the tape to clean and dry skin.
- > Ensure a correct body posture of the patient when applying the tape.
- > Make sure that the cross tape does not arch and that the tape is not creased.
- > Many tapes are stretched during application but ensure that measured force is used. The tissue (circulation) must not be constricted.
- > The MECRON X-patch is a single-use product.

## Indications

- > treatment of trigger points
- > treatment of pain points and acupuncture meridians
- > scar release

#### Contraindications:

- > large and serious skin injuries
- > open wounds with risk of infection
- > nondistinctive diagnoses
- > allergic reactions to tape material

## Technical data

#### Sizes:

2.8 × 2.1 cm, 180 pieces

3.5 × 2.5 cm, 120 pieces

5.2 × 4.5 cm, 40 pieces

**Colour:** Beige

**Material:** Nylon, Acrylic adhesive, Latex free

**Maintenance:** The product does not require maintenance.

## Caution

Always follow these instructions for use. Contact your doctor if you sense any unusual reaction (such as skin swelling, skin redness etc.). In case of improper handling function and maximum safety of the product can no longer be guaranteed. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. This aid is a single-use product.

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TS Co., Ltd.
Emergo Europe B.V.